# ReadyWrap® Calf

Adjustable compression wrap  
Adaptives Kompressionssystem  
Système de compression réglable  
Sistema de compresión ajustable  
Sistema compressivo regolabile

Verstelbaar compressiesysteem  
Adaptívny kompresívny systém  
Uyarlanabilir kompresyon sistemi  
Justerbart kompressionssystem  
الضاغط والقابل للضبط والتعديل

## en Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Calf has adjustable straps with a built-in 50% overlap and muscle support spine to contain the calf. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

**Indications and patient groups**  
■ Lymphedema  
■ Lipedema  
■ Edema  
■ Venous ulcers  
■ Venous insufficiency  
■ Leg fatigue and/or tiredness

### Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema, lipedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

### Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

### Precautions

- ABPI > 1.3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

### Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

### Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

### Donning instructions

Apply ReadyWrap garments in the following sequence:

- toe
- foot
- calf
- thigh
- knee

- Slide the ReadyWrap Liner onto your leg.
- Unfasten all garment straps.
- Align the spine at back of calf with the hook side facing your calf.
- Hold the shortest strap set in opposing hands just above the heel, and place the strap without hook (B) on the front of your leg about 2-3 cm above your ankle bone. While holding that strap down, pull the opposing strap with the long vertical white hook (A) at full stretch and fasten it directly on top of the strap below. (Strap set 1)

on the front of your leg about 2-3 cm above your ankle bone. While holding that strap down, pull the opposing strap with the long vertical white hook (A) at full stretch and fasten it directly on top of the strap below. (Strap set 1)

Fasten the next set of straps by applying the strap with the short horizontal light blue hook (C) at a full stretch onto the straps below. Next, pull the opposing strap with the long vertical light blue hook (D) at full stretch and fasten it directly on top of the strap below. (Strap set 2)

Fasten the remaining straps in sequence using the same process until all straps are in place.

- Regular length calf garments have 5 strap sets. Long length calf garments have 6 strap sets.

### Care instructions

- L&R recommends all ReadyWrap garments be washed and dried before the first extended use.
- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
  - Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

### Material content

Nylon (Polyamide), polyurethane, spandex (elastane)  
Not made with natural rubber latex.

### Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

### Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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