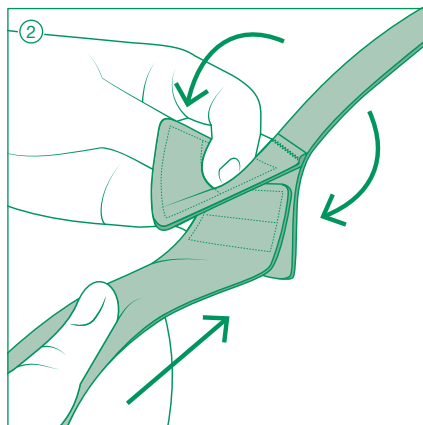
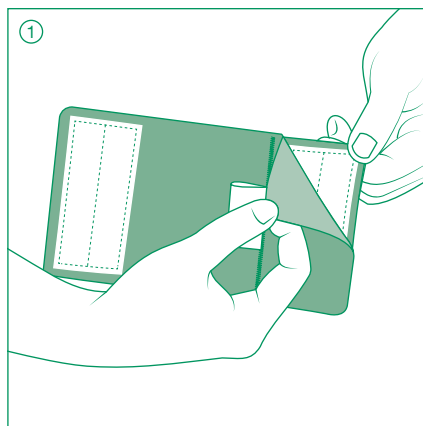
ReadyWrap® Extender Strap

for adjustable compression wrap
zum adaptives Kompressionssystem
pour système de compression ajustable
para sistema de compresión ajustable
per il sistema compressivo regolabile
voor het verstelbaar compressiesysteem
pre adaptívny kompresívny systém
uyarlanabilir kompresyon sistemi için
til det justerbar kompressionssystem

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en Instructions for use

ReadyWrap® Extender Strap is an accessory for use with ReadyWrap Calf, Knee and Thigh adjustable compression wraps which extends the length of a strap 10 cm to accommodate limbs of larger girth or lobules.

Please consult with a trained medical professional prior to first use.

Refer to ReadyWrap garment instructions for use for indications, intended uses, contraindications, and precautions.

Donning instructions

- For use with ReadyWrap Calf, Knee, and Thigh garments.
- Apply the Extender Strap prior to donning the respective garment.

① Unfasten the clamp at the end of the extender strap.

② Place your desired ReadyWrap garment strap (must be a strap with a long, vertical hook) inside of the open end and fasten by closing the hook clamp.

③ Don the ReadyWrap garment according to the garment's instructions.

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Care instructions

L&R recommends all ReadyWrap garments be washed and dried before the first extended use.

- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
 - Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

Material content

Nylon (polyamide), polyurethane, spandex (elastane)

Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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VELCRO® is a registered trademark of Velcro BVBA.
Lohmann & Rauscher AG
L&R UK Medical Ltd.