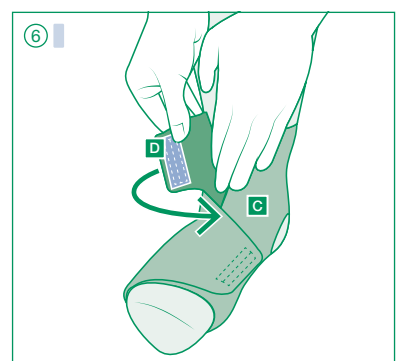
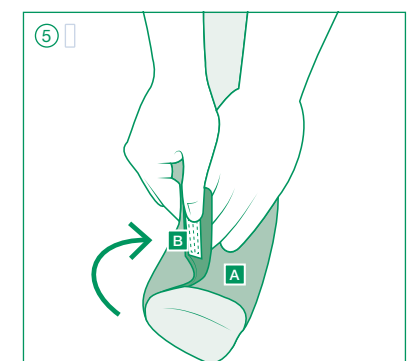
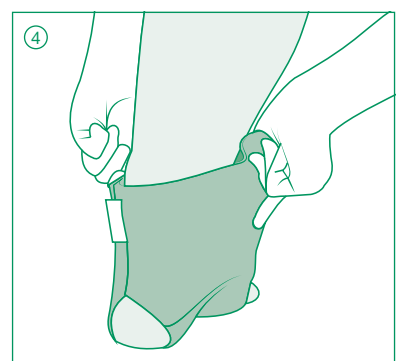
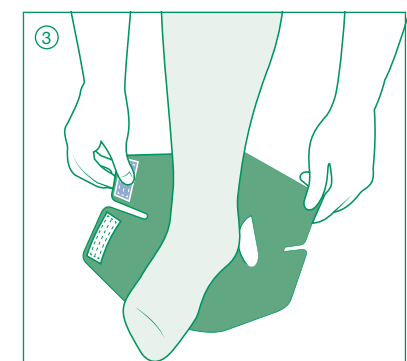
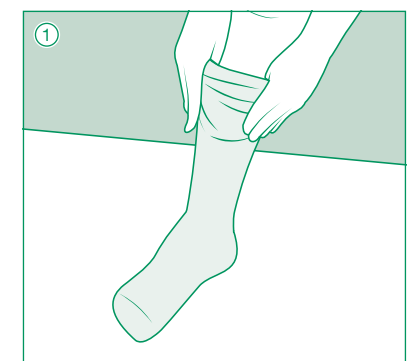
# ReadyWrap<sup>®</sup> Foot

Adjustable compression wrap  
Adaptives Kompressionssystem  
Système de compression réglable  
Sistema de compresión ajustable  
Sistema compressivo regolabile

Verstelbaar compressiesysteem  
Adaptivny kompresivny systém  
Uyarlanabilir kompresyon sistemi  
Justierbar kompressionssystem  
الضاغط والقابل للضبط والتعديل

Symbol glossary provided at: <https://www.lohmann-rauscher.com/label-symbols/>  
VELCRO® is a registered trademark of Velcro BVBA.

**L&R USA INC.**  
3880 W Wheelhouse Road  
Milwaukee, WI 53208 USA  
www.lohmann-rauscher.com

**EMERGO EUROPE**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

**Lohmann & Rauscher AG**  
Oberstrasse 222  
CH-9014 St. Gallen  
Switzerland

**UK Responsible Person**  
L&R UK Medical Ltd,  
1 Wellington Court - Lancaster Park -  
Needwood - Burton upon Trent -  
Staffordshire DE13 9PS -  
United Kingdom

4048618 2632 / 2023-06

## en Instructions for use

Thank you for choosing ReadyWrap<sup>®</sup>!

ReadyWrap is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Foot has adjustable straps to contain the foot and ankle. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

### Indications and patient groups

- Lymphedema
- Lipedema
- Edema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

### Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema, lipedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

### Contraindications

- Arterial insufficiency or ABPI < 0,8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

### Precautions

- ABPI > 1,3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

### Side effects

- In (very) rare cases, skin irritations and/or allergies may occur.

### Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

### Donning instructions

- Apply ReadyWrap garments in the following sequence:
- 1 toe
  - 2 foot
  - 3 calf
  - 4 thigh
  - 5 knee

- ReadyWrap Foot garments are left/right-specific.

- 1 Slide the ReadyWrap Liner onto your leg.

- 2 Unfasten all garment straps.

- 3 Align the garment onto your foot so the strap with white hook is nearest to toes and the strap with light blue hook is at your ankle.

- 4 Place heel into the heel opening. (Garment tag should be above your heel on the garment's exterior.)

- 5 Place the strap closest to toes without the hook **(A)** against the top of your foot. While holding that strap down, pull the opposite strap with white hook **(B)** over top at full stretch and fasten to the top of the garment.

- 6 Place the ankle strap without the hook **(C)** against your ankle. While holding that strap down, pull the strap with light blue hook **(D)** over top at full stretch and fasten to the top of the garment.

### Care instructions

L&R recommends all ReadyWrap garments be washed and dried before the first extended use.

- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
  - Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

### Material content

Nylon (polyamid), polyurethane, spandex (elastane)  
Not made with natural rubber latex.

### Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

### Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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