ReadyWrap® Foot CT

Adjustable compression wrap
Adaptives Kompressionssystem
Système de compression réglable
Sistema de compresión ajustable
Sistema compressivo regolabile

Verstelbaar compressiesysteem
Adaptívny kompresívny systém
Uyarlanabilir kompresyon sistemi
Justerbart kompressionssystem
الضاغط والقابل للضبط والتعديل

en Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Foot CT has adjustable straps to contour the foot and ankle providing the containment and compression needed for moderate to severe indications. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Lymphedema
- Lipedema
- Edema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema, lipedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for plebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

Contraindications

- Arterial insufficiency or ABPI < 0,8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- ABPI > 1,3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

Apply ReadyWrap garments in the following sequence:

- 1 toe
- 2 foot
- 3 calf
- 4 thigh
- 5 knee

- ReadyWrap Foot CT garments are left/right-specific.
- ReadyWrap Foot CT straps have built-in finger grips to make it easier to pull the straps while donning.

- 1 Slide the ReadyWrap Liner onto your leg.

- 2 Unfasten all garment straps. With the colored hook facing up, place your foot inside the garment, aligning your heel in the heel opening (long strap with blue hook will be behind the heel).

- 3 Place the strap closest to your toes without the hook against the top of your foot. While holding that strap down, pull the opposite strap with the two-pronged white hook over top at full stretch and fasten them on top of the strap below. Adjust for fit and comfort.

- 4 Pull the ankle strap with the light blue hook at full stretch across the front of your foot at a downward angle and fasten it to the opposite side. Adjust for fit and comfort.

- 5 Pull the ankle strap with the two-pronged dark blue hook at full stretch around the front of your foot and fasten to the opposite side at a downward angle or aligned with the strap below. Adjust for fit and comfort.

Care instructions

- L&R recommends all ReadyWrap garments be washed and dried before the first extended use.
- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
 - ◆ Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

Material content

Nylon (polyamide), polyurethane, spandex (elastane)
Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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