# ReadyWrap® Foot SL

Adjustable compression wrap  
Adaptives Kompressionssystem  
Système de compression réglable  
Sistema de compresión ajustable  
Sistema compressivo regolabile

Verstelbaar compressiesysteem  
Adaptivny kompresivny systém  
Uyarlanabilir kompresyon sistemi  
Justerbart kompressionssystem  
الضاغط والقابل للضغط والتعديل

## en Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Foot SL contains the top of the foot, and has adjustable straps to contour around the ankle. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

### Indications and patient groups

- Lymphedema
- Lipedema
- Edema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

### Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema, lipedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

### Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

### Precautions

- ABPI > 1.3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

### Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

### Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

### Donning instructions

- 1 toe
- 2 foot
- 3 calf
- 4 thigh
- 5 knee

■ ReadyWrap Foot SL garments are left/right-specific.

① Slide the ReadyWrap Liner onto your foot and pull it up the leg.

② Unfasten all garment straps.

③ Slide the garment onto your foot with the hook receptive side on top.

④ Pull the strap with the white hook at full stretch around the back and fasten it to the side or top of the garment based on fit and comfort.

⑤ Pull the strap with the light blue hook at full stretch around the front and fasten it to the garment at a downward angle or aligned with the strap below based on fit and full coverage.

### Care instructions

- L&R recommends all ReadyWrap garments be washed and dried before the first extended use.
- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
  - ◆ Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

### Material content

Nylon (polyamide), polyester, polyurethane, spandex (elastane)  
Not made with natural rubber latex.

### Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

### Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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#### AT Lohmann & Rauscher GmbH
#### SV Lohmann & Rauscher AB
#### CH (CH-REP) Lohmann & Rauscher AG