



People.Health.Care.

Suprasorb® F Protect Film Wound Dressing.



Flexible & elastic



- conforms to challenging body contours
- stretches with the skin for more patient comfort
- secure grip for high wearing comfort

Easy to use



- can be readily applied with gloves
- non-touch application
- transparency allows monitoring of the wound

Protects wounds




- water- and bacteriaproof
- water vapour permeability maintains a moist wound environment
- reduction of friction-related skin damage

Suprasorb® F Protect

Mode of action:

- A** transparent polyurethane film
- B** water- and bacteria-proof
- C** excess water vapour evaporates

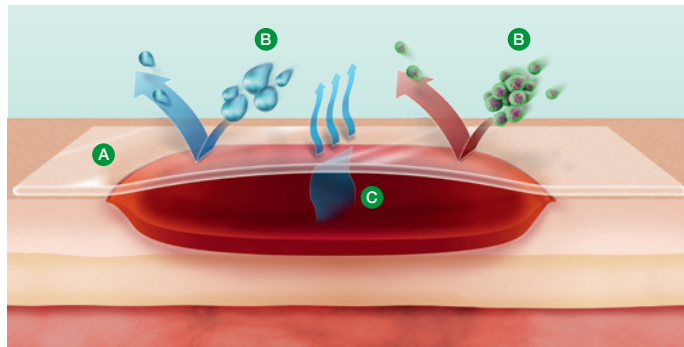

Indications:

- arterial and venous ulcers
- diabetic ulcers
- pressure ulcers (also suitable for prophylaxis)
- skin graft donor sites
- surgical wounds
- abrasions
- lacerations and cuts
- minor burns/scalds
(1st degree/2nd degree superficial)

Can be used as a secondary dressing for wounds at risk of infection.

Wound depth:



Superficial wounds


Level of exudate:



Lightly exuding wounds

Important information:

- can be used as a secondary dressing for fixation
- to protect new epithelial tissue
- the product is suitable for showering
- resistant to common disinfectants
- this product is free from colophony, colophony derivatives and DMF

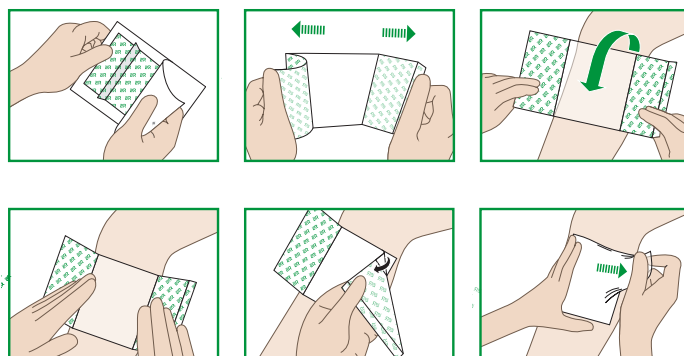

Instructions for use:

- cleanse the wound and pat the edges dry **before applying the dressing** – Suprasorb F Protect adheres best to dry skin
- the **dressing can be applied** easily and quickly using the non-touch application technique

Suprasorb® F Protect sterile film wound dressing

Size (cm)	REF	Shipping unit (pcs.)
5 x 7	162 460	10/380
5 x 7	162 461	100/1,000
10 x 12	162 462	10/100
10 x 12	162 463	50/200
10 x 25	162 464	10/140
15 x 20	162 465	10/400
20 x 30	162 466	10/180