




People.Health.Care.

Discover the new side of Debrisoft® – **Debrisoft® Duo.**

When I want
to improve, I

debrisoft it.

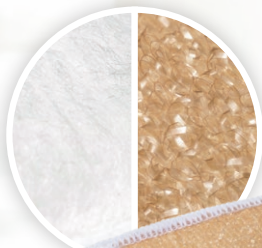

Two sides,
two textures,
one solution.



When simplicity matters, I

debrisoft it.



Efficient 2-in-1 debridement pad:

- the **original soft, white side** removes debris, exudate and biofilm from the wound
- the **new textured beige side** makes it easy to loosen firmly adherent, fibrinous slough*

Developed based on user experience to allow more effective debridement of firmly adhering, fibrinous slough than standard mechanical debridement.

Debrisoft® Duo would be recommended by 100% of users in an acceptance test.

Debrisoft® Duo

Simplify your debridement routine.



1. Moisten

Moisten both sides of Debrisoft Duo before usage with saline or wound irrigation solution.



2. Adapt to needs

Start with either side of Debrisoft Duo, depending on the wound condition and switch as needed.



3. Debride

Applying gentle pressure, wipe the pad over the wound or the affected skin area for 2–4 minutes, until devitalised tissue and debris are removed.



4. Done

Dispose all Debrisoft products after use.

When I need rapid results, I
debrisoft it.

Debrisoft® Duo
10 x 10 cm

- designed for debridement of fibrinous slough and challenging wounds
- the textured beige side makes it easy to loosen firmly adherent fibrinous slough
- the soft white side removes debris, exudate and biofilm from the wound
- you have the flexibility to use either side of the pad depending on the wound condition and patient need

Debrisoft® Lolly

- for debridement of deep wounds and hard-to-reach areas
- secure fixation of the fibre head (low risk by sudden release of the head)
- ideal for surgical invasive wounds


Debrisoft® Duo
13 x 20 cm



Designed based on user experience:

Highly effective

Absorbs, binds and removes debris, exudate and bacteria. Removes 99% of biofilm*.

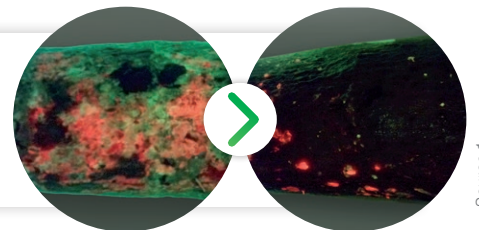

Source 1

Easy and safe to use

High user satisfaction, rapidly visible results within 2–4 minutes.



Source 2

Adapt to wound situation and patient needs

Debrisoft Duo was developed to enable more effective debridement of firmly adhering, fibrinous slough than standard mechanical debridement.





The Debrisoft® product family

Three options – one solution.



Want to use the original?
Get helpful tips here ...



Debrisoft® Duo

sterile, individually sealed

Size (cm)	REF	Shipping units
10 x 10	159 990	5/50
13 x 20	159 991	5/50

Debrisoft® Lolly

sterile, individually sealed

REF	Shipping units
33 224	5/50

Please note the instructions for use.

Sources:

- 1) Images: Photos used with patient's kind consent, UK.
- 2) Images: Gabriele Stern, ATZ-Stern, Bocholt, Germany.

Try Debrisoft® Duo now!
For more flexibility
in debridement.

