


Suprasorb®

CNP EasyDress

The benefits at a glance.

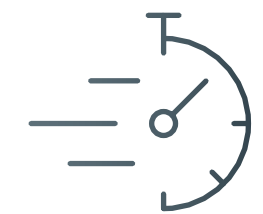

EasyDress can be easily applied and quickly secured and it forms a reliable seal. EasyDress is therefore both **time-saving and cost-effective**.



EasyDress is airtight and impermeable to microorganisms, but **is also highly permeable to water vapour** (min. 2000 ml/m² in 24 hours) and **non-adhesive**. This secondary dressing is therefore gentle on the patient's intact surrounding skin.



The **special transparent film** allows the primary dressing and surrounding skin to be observed.




EasyDress is available in three sizes and **can be cut to any required length**. The EasyDress Fixateur Externe is the 4th dimension in the EasyDress range.

90%

Up to 90% time savings when applying the dressing.

92%

92% of users would use EasyDress again immediately.¹

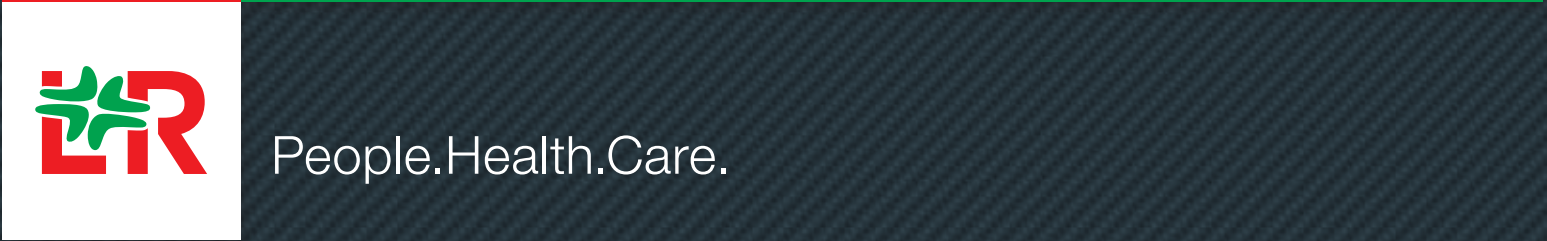


Suprasorb® CNP EasyDress

REF	Size	Tubular bandage			Retention strips
		1	2	3	10 cm x 12 cm
34901	S	50 cm	30 cm	70 cm	5 pcs.
34902	M	65 cm	40 cm	85 cm	7 pcs.
34903	L	80 cm	50 cm	95 cm	8 pcs.

The 4th dimension of the EasyDress range:
Suprasorb® CNP EasyDress Fixateur Externe

REF	Size	Tubular bandage		Retention strips
		1	2	10 cm x 25 cm
34905	One Size	150 cm	120 cm	6 pcs.

Source:
1) M. Duft, D. Ostapowicz, S. de Lange, K. Griefßhammer, M. Abel: Audit on the usability, user and patient satisfaction of tubular bandage, Multi-center international post-marketing clinical study under real conditions.

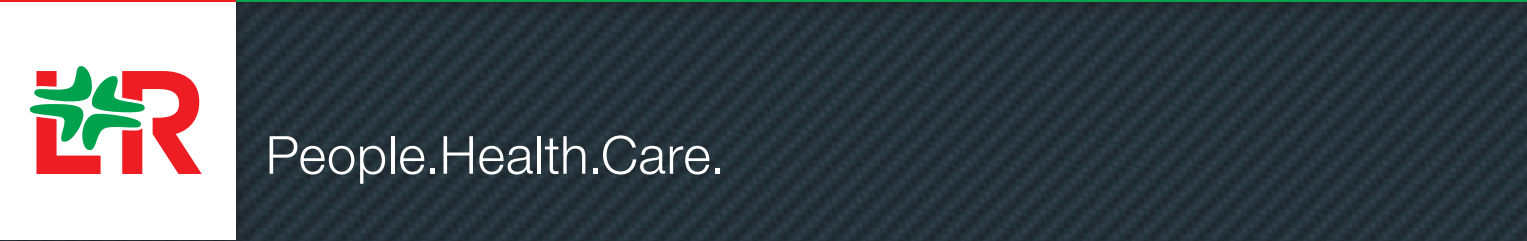

Suprasorb®

CNP EasyDress

3D innovation for negative pressure therapy on the extremities.



EasyDress



Patented. Awarded a prize for innovation.


Suprasorb®

CNP EasyDress

Our secondary dressing
for all your therapy needs.

Pull over – secure – start therapy. That's how easy EasyDress is. Thanks to the 3D design and the flexible special material, this secondary dressing for all your negative pressure therapy needs becomes airtight **considerably more quickly than other conventional adhesive methods.**

EasyDress can be applied easily without assistance. It is available in different sizes and can be cut to the required length – making it very easy to adjust to fit the different anatomies of the upper and lower extremities. Not to mention, **it can be used to treat even difficult-to-reach wound sites.**

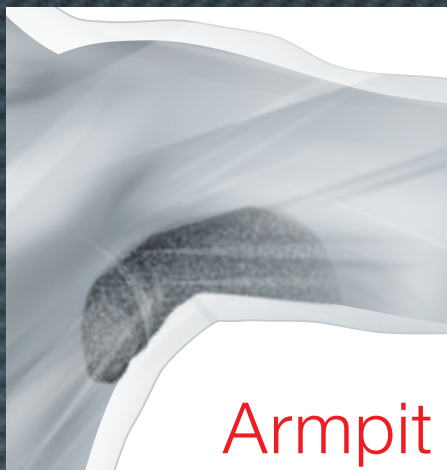


Significantly better feedback.

EasyDress impresses with its excellent wearing comfort and protects the intact surrounding skin. Its comfort and functionality result in significantly better feedback than with conventional products, and thus result in higher user acceptance.

Suprasorb®

CNP EasyDress


Easy, fast and gentle to the skin.

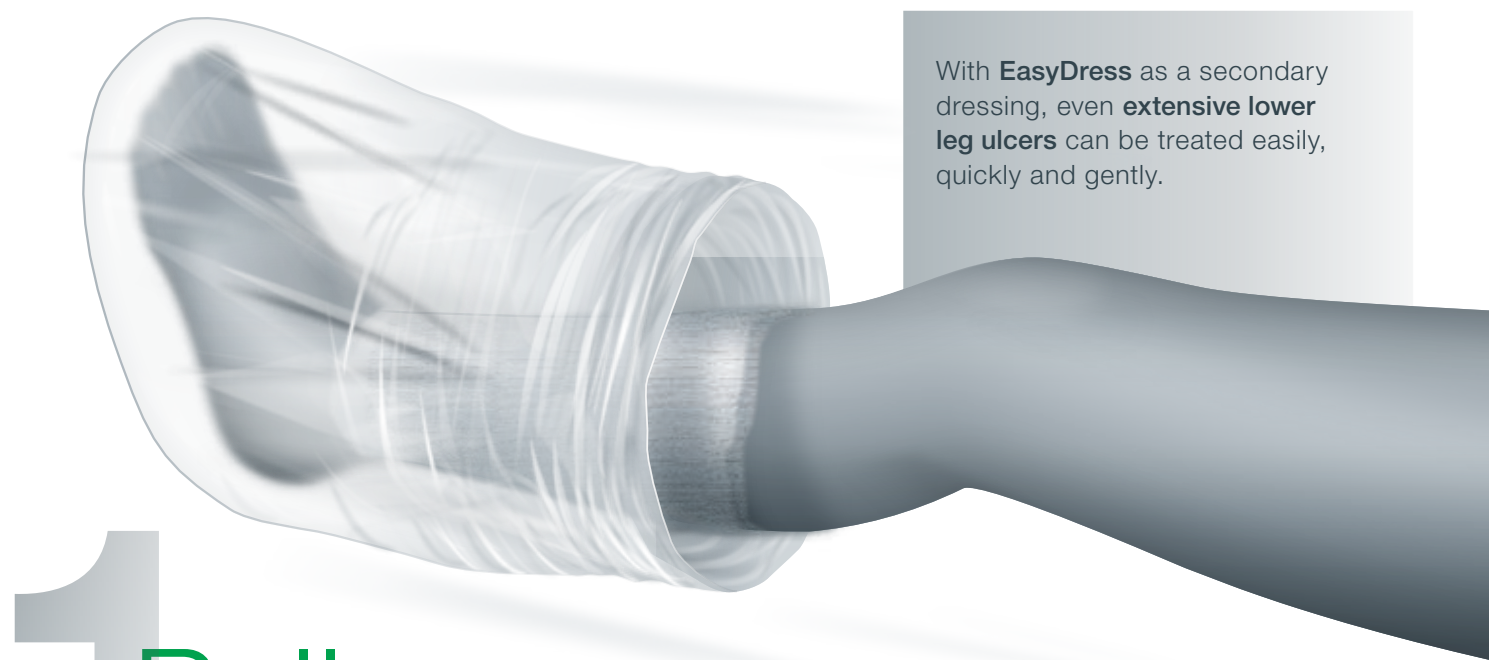

With **EasyDress** as a secondary dressing, even **extensive lower leg ulcers** can be treated easily, quickly and gently.

1 Pull over

Select the correct size of **EasyDress**, cut to size if required and pull over the extremity to be treated. You have already treated the wound with a wound filler of your choice, such as antimicrobial gauze or wound foam.

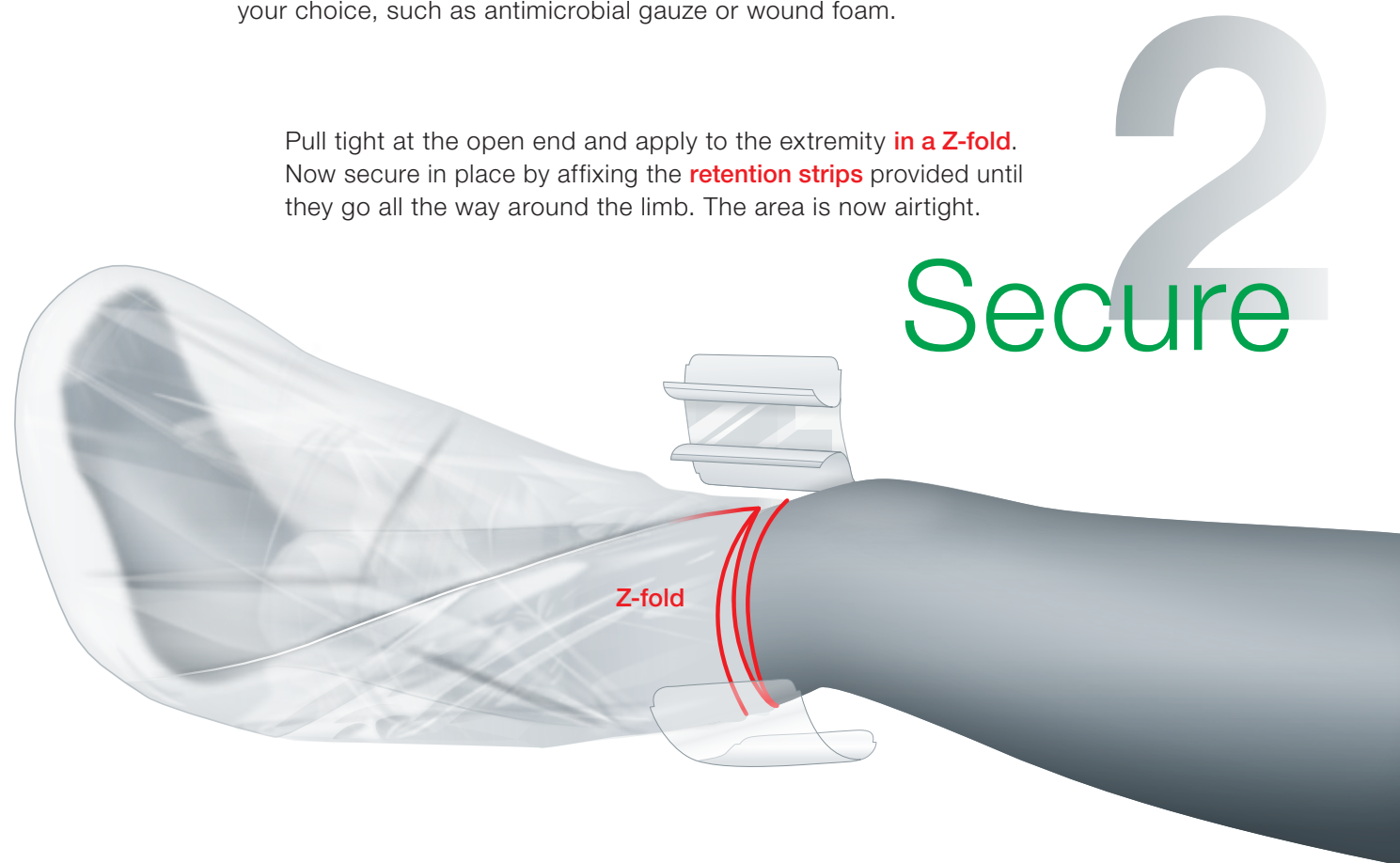

Pull tight at the open end and apply to the extremity **in a Z-fold**. Now secure in place by affixing the **retention strips** provided until they go all the way around the limb. The area is now airtight.

2 Secure



3 Start therapy

Place the drainage pad over the hole cut in the EasyDress dressing above the wound site and connect it to the CNP device. **All set! The negative pressure therapy can start.**

Universal – for use on arms and legs

EasyDress adapts to the **different anatomies** of the upper and lower extremities.

