






TributeWrap™ Glove

Adjustable variable compression foam
garment for lymphedema and edema

EN

TributeWrap™ Glove
Adjustable variable compression foam
garment for lymphedema and edema
Instructions for Use

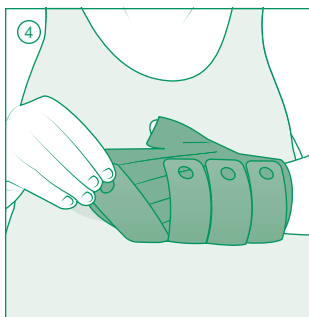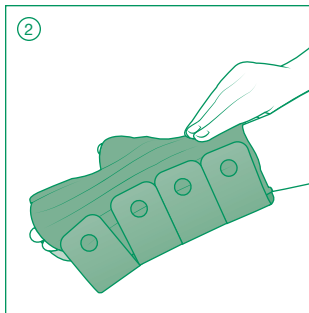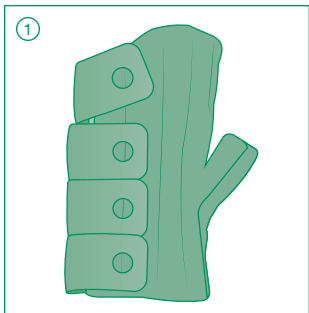

Thank you for choosing TributeWrap™!

TributeWrap™ is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the hand during the evening, night, or low activity. TributeWrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

EN

Indications

- Various stages of lymphedema
- Edema

Intended Uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (e.g. cellulitis)
- Allergies or sensitivities to materials

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Donning instructions

- ① **Fasten** the narrow straps at the edge of the glove.
- ② **Slide** your hand into the glove ensuring your fingers align in the individual spacers and thumb in separate thumb space.
- ③ **Pull** the three straps closest to your wrist using the finger grip and fasten to the garment.
- ④ **Pull** the last strap and fasten to the garment at an angle. Adjust all straps for fit and comfort.
- ⑤ **Slide** the Sleep Sleeve over the glove leaving 1" at both ends of the glove and fasten.

Material content

Blend of: nylon (polyamide), polyurethane, polyester, spandex (elastane)

Not made with natural rubber latex.

Care instructions

L&R USA recommends all TributeWrap garments be washed and dried before the first extended use.



Roll back and fasten straps to garment prior to laundering.

Washing

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean

Drying

- Tumble dry low heat with other laundry items
- Do not iron
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls



Warranty

We guarantee your TributeWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.



Symbol glossary provided at:
www.lohmann-rauscher.us/symbolsglossary



L&R USA INC.

3880 W Wheelhouse Road, Milwaukee, WI 53208 USA
www.Lohmann-Rauscher.us | 1-855-892-4140



EMERGO EUROPE

Prinsessegracht 20, 2514 AP The Hague, The Netherlands