







# TributeWrap™

## Below Knee

Adjustable variable compression foam garment for lymphedema and edema

EN

**TributeWrap™ Below Knee**  
**Adjustable variable compression foam**  
**garment for lymphedema and edema**  
Instructions for Use

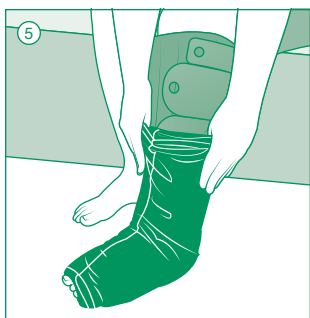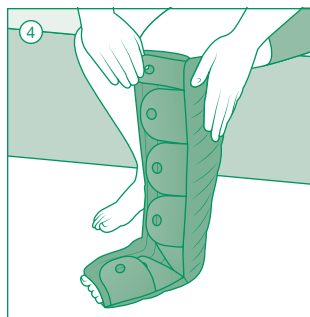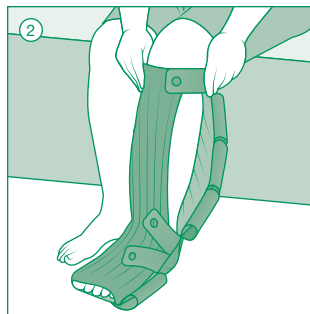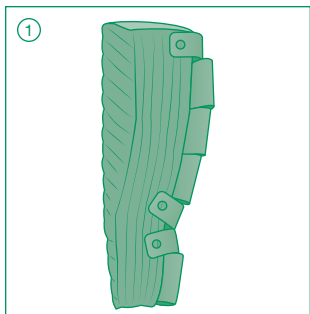

# Thank you for choosing TributeWrap™!

TributeWrap™ Below Knee is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the lower leg and foot during the evening, night, or low activity. TributeWrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

EN

## Indications

- Various stages of lymphedema
- Edema

## Intended Uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

## Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (e.g. cellulitis)
- Allergies or sensitivities to materials

## Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

## Donning instructions

- ① **Fasten** the narrow straps at the edge of the garment and loosely roll the wide straps back.
- ② **Slide** your foot into the wide end of the garment and pull it up your leg.
- ③ **Pull** the two narrow straps at your ankle using the finger grip and fasten to the garment at an angle.
- ④ **Fasten** the remaining straps using the same process. Adjust all straps for fit and comfort.
- ⑤ **Slide** the wide end of the Sleep Sleeve over the garment and fasten it 2" below the top of garment.

## Material content

Blend of: nylon (polyamide), polyurethane, polyester, spandex (elastane)

Not made with natural rubber latex.

## Care instructions

L&R USA recommends all TributeWrap garments be washed and dried before the first extended use.



Roll back and fasten straps to garment prior to laundering.

## Washing

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean

## Drying

- Tumble dry low heat with other laundry items
- Do not iron
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls



## Warranty

We guarantee your TributeWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.



Symbol glossary provided at:  
[www.lohmann-rauscher.us/symbolsglossary](http://www.lohmann-rauscher.us/symbolsglossary)



**L&R USA INC.**

3880 W Wheelhouse Road, Milwaukee, WI 53208 USA  
[www.Lohmann-Rauscher.us](http://www.Lohmann-Rauscher.us) | 1-855-892-4140



**EMERGO EUROPE**

Prinsessegracht 20, 2514 AP The Hague, The Netherlands