



People.Health.Care.

# BIOCOL™

## Deliver collagen precisely where it's needed.

BIOCOL™ is a bovine collagen powder which interacts with the wound forming a gel that provides a moist wound healing environment, reduces inflammation and promotes cellular re-generation.

### Benefits

- Advanced accordion dispenser delivers collagen exactly where it is needed
- 100% modified collagen
  - ◆ Prepared from 90% Type I collagen content
- Supplies nutritive proteins and amino acids directly to the wound site
- Easily reaches deep into tunneling or undermined wounds

### Indications

- Chronic and acute wounds
- Full and Partial-thickness wounds, including:
  - ◆ Venous stasis ulcers
  - ◆ Pressure ulcers
  - ◆ Diabetic ulcers
  - ◆ Arterial ulcers
  - ◆ 1<sup>st</sup> and 2<sup>nd</sup> degree burns

### Application

1. Prepare the wound site using standard wound bed preparation protocol.
  2. Push and bend dispenser top to open and apply BIOCOL to the wound site. Apply powder over the entire wound surface up to 1/8th inch thick loosely filling cavities or undermined areas.
  3. Cover the wound with a secondary dressing.
  4. Maintain a moist wound healing environment.
    - ◆ Change the secondary dressing and reapply BIOCOL as required.
- Note the BIOCOL collagen powder need not be removed from the wound site. The collagen powder is readily broken down by the natural enzymes present in wound fluids.



### BIOCOL™

in precision applicator, sterile

Contents	HCPCS Code	Item No.	Shipping Units (per box/case)
1 g	A6010	154892	15/120
5 g	A6010	154893	15/120

