Caresia®

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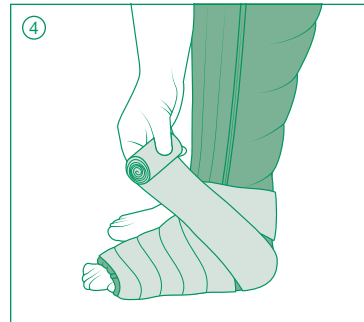
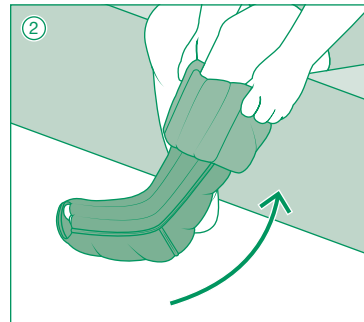
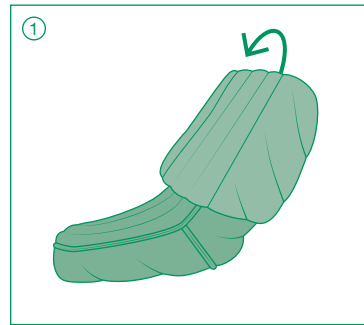
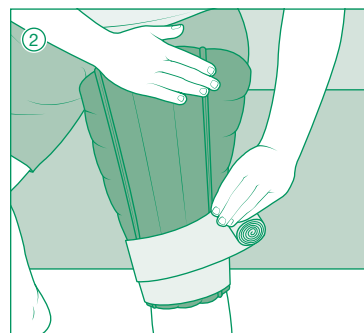
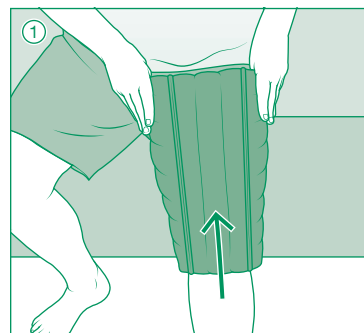
en | Instructions for use

Thank you for choosing Caresia®!

Caresia® Foot, Below Knee, and Thigh garments are off-the-shelf foam bandage liners for lymphedema and edema management of the foot and/or leg. Caresia can be used day or night for the intensive phase of therapy or in the self-management phase as recommended by your medical professional.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended uses

- Assists in softening of indurated/fibrotic tissue (when used with compression bandages/compression garments)
- Fill concavities and/or pad bony prominences (when used with compression bandages/compression garments)
- Diffuse pressure points (when used with compression bandages/compression garments)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

Foot/Below Knee

If pairing a Below Knee and Thigh garment, apply Thigh garment first.

- 1 Fold the upper portion of the garment inside out overlapping the lower portion.
- 2 Slide the folded garment over the foot and heel.
- 3 Unfold the garment onto your leg. Adjust for fit and comfort.

- If pairing a Below Knee and Thigh garment, unfold the Below Knee garment to overlap the knee portion of the Thigh garment.

- 4 Apply short stretch bandages over top as directed by your medical professional.

Thigh

- 1 Slide the garment over your foot onto your knee and thigh ensuring the outer seams are at the front of the garment on both sides of your leg.

- 2 Apply short stretch bandages over top as directed by your medical professional.

Care instructions

L&R USA recommends all Caresia garments be washed and dried before the first extended use.

Washing

- Machine wash with mild detergent on permanent press
- Do not use fabric softener, bleach, or other additives
- Do not dry clean

Drying

- Tumble dry on permanent press
- Do not iron

Material content

Polyurethane, nylon (polyamide), spandex (elastane)
Not made with natural rubber latex.

Warranty

We guarantee your Caresia garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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