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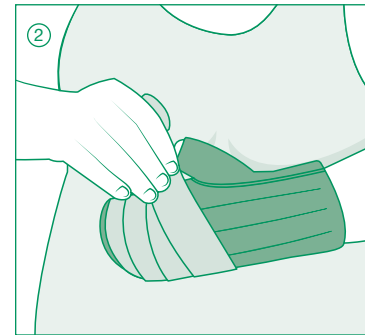
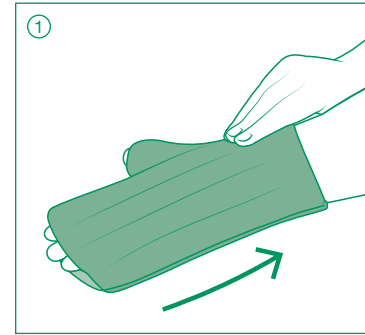
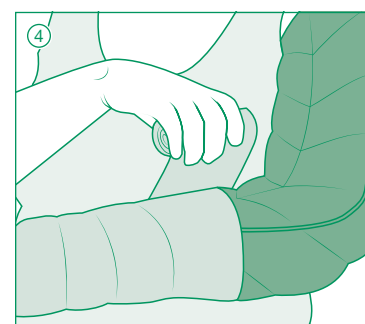
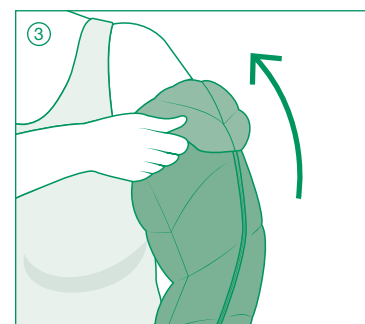
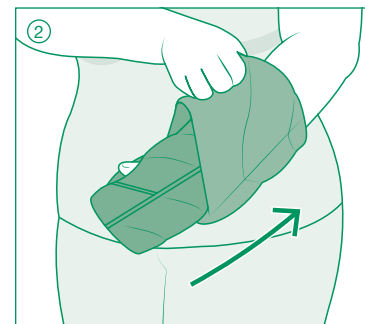
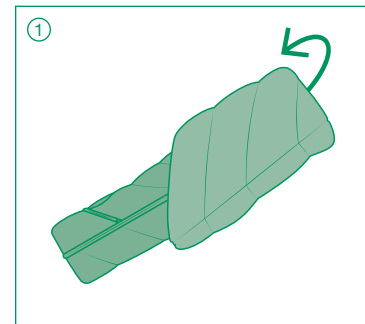
en | Instructions for use

Thank you for choosing Caresia®!

Caresia® Glove, Gauntlet, Wrist to Axilla, and MCP to Axilla garments are off-the-shelf foam bandage liners for lymphedema and edema management of the hand and/or arm. Caresia can be used day or night for the intensive phase of therapy or in the self-management phase as recommended by your medical professional.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended uses

- Assists in softening of indurated/fibrotic tissue (when used with compression bandages/compression garments)
- Fill concavities and/or pad bony prominences (when used with compression bandages/compression garments)
- Diffuse pressure points (when used with compression bandages/compression garments)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

Glove/Gauntlet

- Slide the Glove/Gauntlet over your hand ensuring your fingers are properly placed in digit spacers and your thumb in the separate thumb space.
- Apply short stretch bandages over top as directed by your medical professional.

Wrist to Axilla/MCP to Axilla

- Fold the upper portion of the garment inside out overlapping the lower portion.
- Slide the folded garment over your hand* and forearm ensuring the outer seam is on the top of your arm.
 - For MCP to Axilla style, ensure your thumb is positioned in the thumb space.
- Unfold the garment onto your upper arm. Adjust for fit and comfort.
- Apply short stretch bandages over top as directed by your medical professional.

Care instructions

L&R USA recommends all Caresia garments be washed and dried before the first extended use.

Washing

- Machine wash with mild detergent on permanent press
- Do not use fabric softener, bleach, or other additives
- Do not dry clean

Drying

- Tumble dry on permanent press
- Do not iron

Material content

Polyurethane, nylon (polyamide), spandex (elastane)
 Not made with natural rubber latex.

Warranty

We guarantee your Caresia garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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