en Instructions for use for ReadyWrap® Arm adjustable compression wrap

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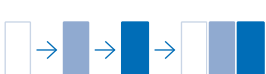
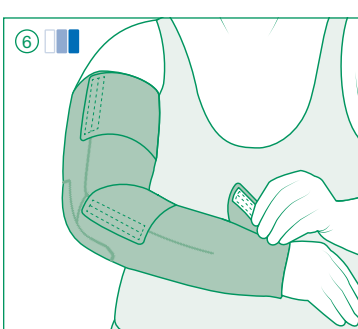
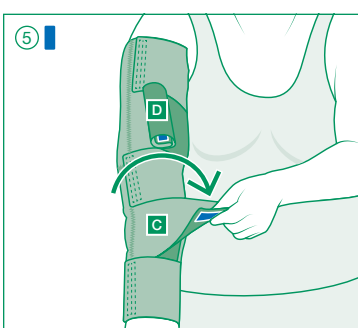
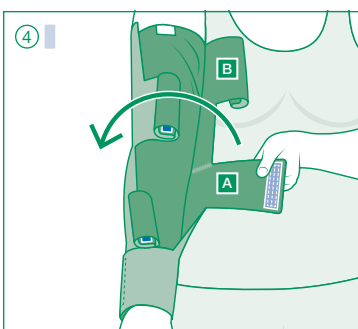
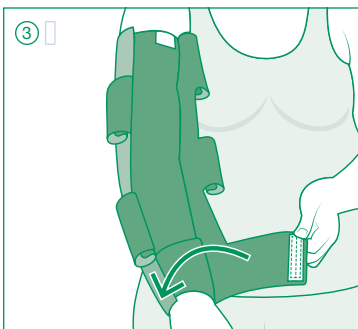
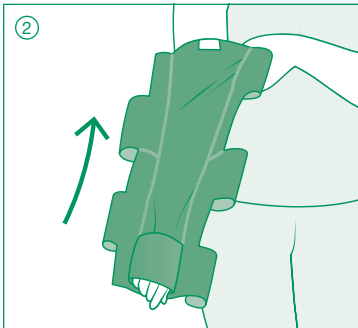
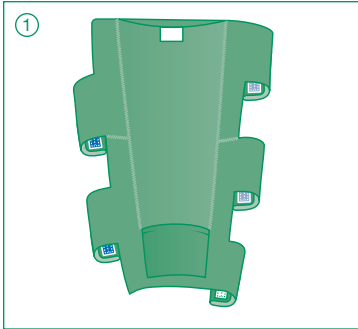
en | Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap® is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Arm has adjustable straps to contain the arm while allowing one-handed donning. It has foam at both the wrist and elbow to pad the bony prominences. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

Indications

- Lymphedema
- Edema

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema and edema
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- ① **Unfasten** and roll back all garment straps.
- ② **Slide** your arm into the built-in liner through the wide end (with garment tag) until it is properly aligned from the wrist to upper arm.
- ③ Pull the strap at the wrist with the white hook and lightly fasten it to the exterior of the garment on the opposing side.
- ④ Pull the next two straps with the light blue hook at the forearm **(A)** and upper arm **(B)**, one at a time and lightly fasten them to the exterior of the garment on the opposing side.
- ⑤ Pull the final two straps with the dark blue hook **(C, D)**, one at a time and lightly fasten them to the exterior of the garment on the opposing side.
- ⑥ **Re-adjust** each strap by pulling them at a full stretch and fastening them securely to the exterior of the garment; beginning with the wrist, then continuing sequentially up the arm.

Care instructions

L&R USA recommends all ReadyWrap garments be washed and dried before the first extended use.

Washing

- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean

Drying

- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
 - Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

Material content

Nylon (polyamide), polyurethane, spandex (elastane)

Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

Apply ReadyWrap garments in the following sequence:

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VELCRO® is a registered trademark of Velcro BVBA.