en Instructions for use for ReadyWrap® Gauntlet adjustable compression wrap

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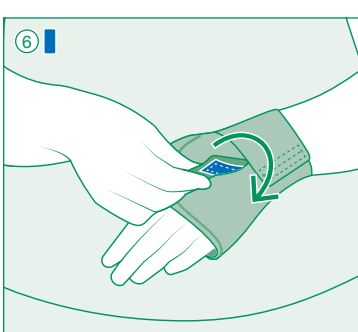
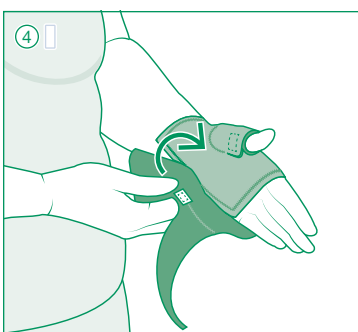
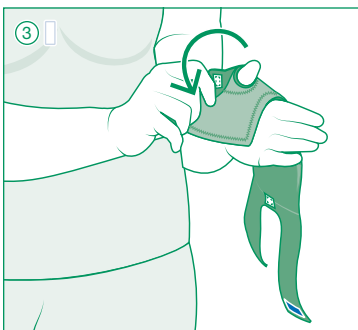
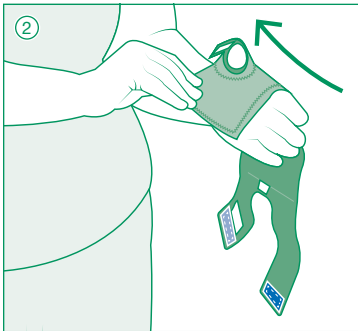
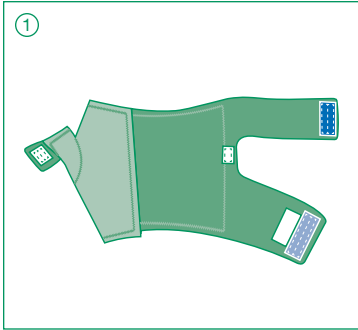
en | Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap® is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Gauntlet has adjustable straps to address the top of the hand. It also offers an optional pocket to insert a foam pad. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

Indications

- Lymphedema
- Edema

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema and edema
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- ReadyWrap Gauntlet garments are left/right-specific.

① Unfasten all garment straps.

② Slide your hand into the gauntlet with your thumb in thumb space and palm facing away from the garment.

③ Fasten the small thumb strap with white hook.

④ Fasten the small strap with the white hook on the palm of the hand to close the garment.

⑤ Pull the strap with the light blue hook at full stretch ensuring it is aligned with the opposing side of the gauntlet and fasten it on the garment.

⑥ Pull the strap with the dark blue hook at full stretch and fasten it on the garment.

Care instructions

L&R USA recommends all ReadyWrap garments be washed and dried before the first extended use.

Washing

- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean

Drying

- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
 - ◆ Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

Material content

Nylon (polyamide), polyurethane, spandex (elastane)

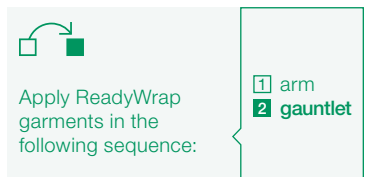
Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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VELCRO® is a registered trademark of Velcro BVBA.
Lohmann & Rauscher A/S
L&R Medical UK Ltd.