en Instructions for use for ReadyWrap® Thigh adjustable compression wrap

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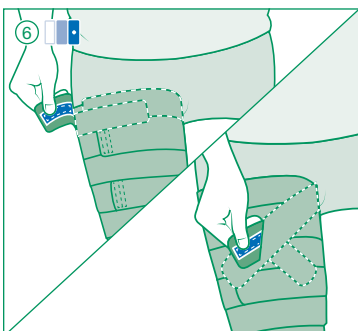
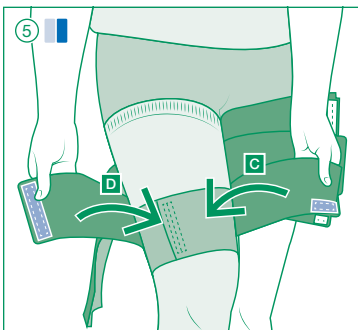
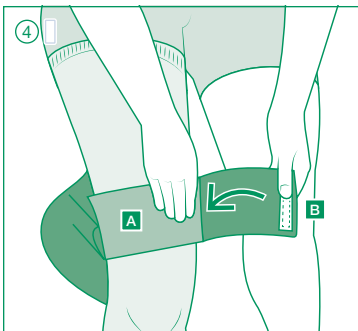
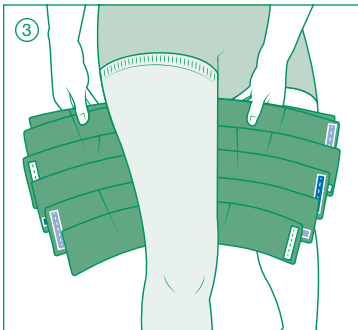
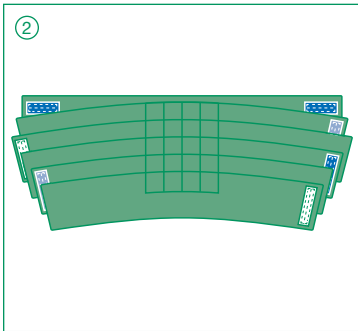
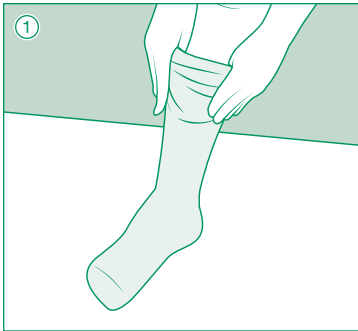
en | Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap® is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Thigh has adjustable straps with a built-in 50% overlap and muscle support spine to contain the thigh. Its top two narrow straps can be adjusted to accommodate the presentation of your thigh. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

Indications

- Lymphedema
- Edema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Material content
Nylon (polyamide), polyurethane, spandex (elastane)

Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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VELCRO® is a registered trademark of Velcro BVBA.