en Instructions for use for ReadyWrap® Toe adjustable compression wrap

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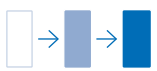
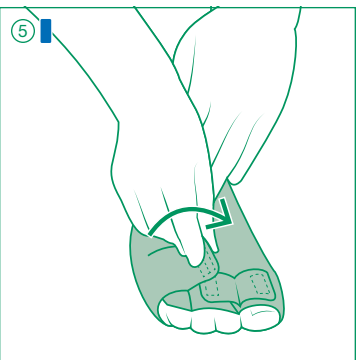
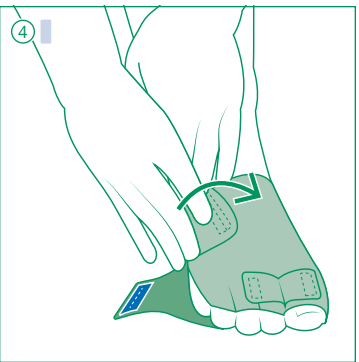
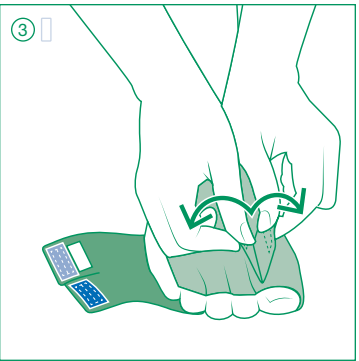
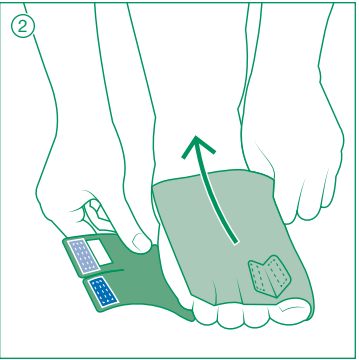
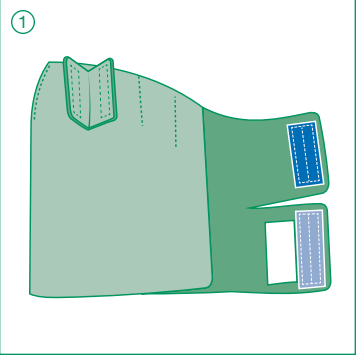
en | Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap® is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Toe has built-in toe spacers and adjustable straps to contain the toes and part of the foot. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

- Indications**
- Lymphedema
 - Edema
 - Venous ulcers
 - Venous insufficiency
 - Leg fatigue and/or tiredness

- Intended uses**
- To maintain reduced fluid volume and shape of limbs
 - Management of lymphedema and edema
 - Relief of leg fatigue and heaviness
 - Assist in management of some symptoms of venous ulcers
 - Assists in helping limit venous ulcer recurrence
 - Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

- Contraindications**
- Arterial insufficiency or ABPI < 0.8
 - Moderate to severe peripheral arterial disease
 - Acute DVT
 - Untreated congestive heart failure
 - Untreated cancer
 - Untreated infection (i.e. cellulitis)
 - Known allergy and/or hypersensitivity to any of the product components
 - Severe cognitive impairment

- Precautions**
- Absent or impaired sensation (peripheral neuropathy)
 - Any open wounds should be dressed prior to application
 - Cardiac overload is suspected
 - Patients have diabetes
 - Patients have advanced small vessel disease
 - Arterial disease is present
 - Renal failure is present
 - Rheumatoid arthritis is present

Side effects
In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident
In case of serious incident, contact the manufacturer and the responsible health authorities.

- Donning instructions**
- ReadyWrap Toe garments are left/right-specific.
 - ReadyWrap Liners cannot be worn with Toe garments due to the individual toe spacers.

- ① **Unfasten** all garment straps.
- ② **Slide** onto your foot aligning the toes in the toe spacers.
- ③ **Pull** the two toe straps with white hook at full stretch simultaneously in opposing directions and fasten to the top of the garment.
- ④ **Pull** the strap with light blue hook at full stretch and fasten it to the top of the garment.
- ⑤ **Pull** the strap with dark blue hook at full stretch and fasten it to the top of the garment.

Care instructions
L&R USA recommends all ReadyWrap garments be washed and dried before the first extended use.

- Washing**
- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
 - Do not use fabric softener or bleach
 - Do not dry clean

- Drying**
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
 - ◆ Be sure that the VELCRO® brand fasteners do not contact your towel
 - Do not iron

Material content
Nylon (polyamide), polyurethane, spandex (elastane)

Not made with natural rubber latex.

Warranty
We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal
Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

Apply ReadyWrap garments in the following sequence:

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VELCRO® is a registered trademark of Velcro BVBA.