en Instructions for use for SwellSpots® Edema and fibrosis pad

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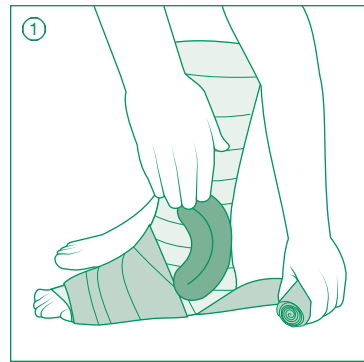
en | Instructions for use

Thank you for choosing SwellSpots®!

SwellSpots® are professionally designed edema and fibrosis pads available in a wide range of versatile shapes and sizes that tuck into compression garments or under compression bandages.

Please consult a trained medical professional prior to first use.

Indications

- Lymphedema
- Edema

Intended uses

- Assists in softening of indurated/fibrotic tissue (when used with compression bandages/compression garments)
- Fill concavities and/or pad bony prominences (when used with compression bandages/compression garments)
- Diffuse pressure points (when used with compression bandages/compression garments)

Contraindications

No contraindications known.

Precautions

- Allergies or sensitivities to materials
- Care should be taken in patients using anticoagulant medications to assess tolerance

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

SwellSpots Pads are versatile pads that can be used in various locations on the body.

Please follow the instructions below and consult your medical professional for how they would like the SwellSpots Pad to be used.

- ① Place either side of the pad on the area identified by your medical professional.
 - ◆ Can be worn directly on the skin
- ② Apply compression bandages or compression garments over the pad as directed by your medical professional.

Care instructions

L&R USA recommends all SwellSpots pads be washed and dried before the first extended use.

SwellSpots pads can be washed and dried with your other laundry.

Washing

- Machine wash with mild detergent on permanent press
- Do not use fabric softener, bleach, or other additives
- Do not dry clean

Drying

- Tumble dry on permanent press
- Do not iron

Material content

Polyurethane, polyester, nylon (polyamide), spandex (elastane)
Not made with natural rubber latex.

Warranty

We guarantee your SwellSpots pad to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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Lohmann & Rauscher B.V
Lohmann & Rauscher Limited