en Instructions for use for Tribute® Wrap Below Knee Adjustable foam compression wrap

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




Tribute® Wrap Below Knee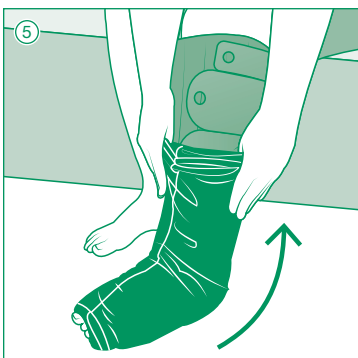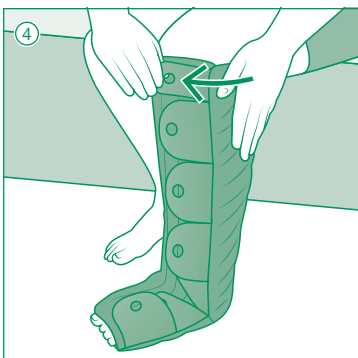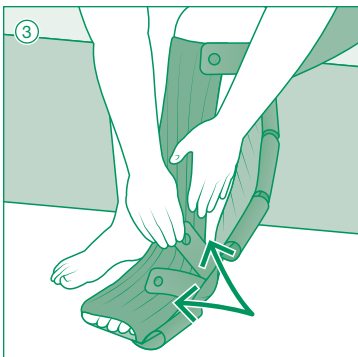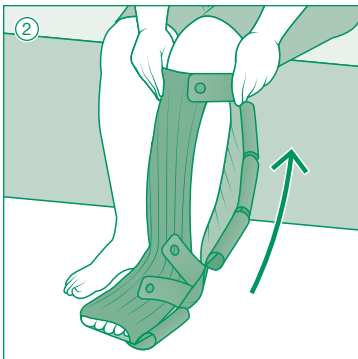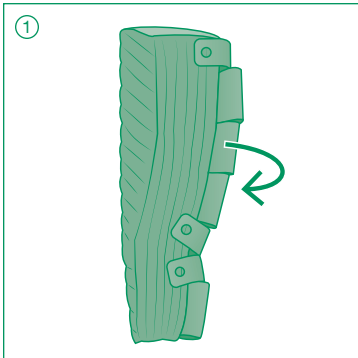

en | Instructions for use

Thank you for choosing Tribute® Wrap!

Tribute® Wrap Below Knee is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the lower leg and foot during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.


Donning instructions

- 1 **Fasten** the two narrow ankle straps and the final strap closest to your knee at the edge of the garment. Roll the remaining straps back loosely.
- 2 **Slide** your foot into the wide end of the garment and pull it up your leg.
- 3 **Pull** the two narrow ankle straps using the finger grip and fasten to the garment at an angle. Adjust for fit and comfort.
- 4 **Fasten** the remaining straps using the same process.
- 5 **Slide** the wide end of the Sleep Sleeve over the garment and fasten it 2" below the top of garment.

Note: Illustration shown with Regular length garment. Other garment lengths have slightly different strap layout.

Care instructions

L&R USA recommends all Tribute Wrap garments be washed and dried before the first extended use.

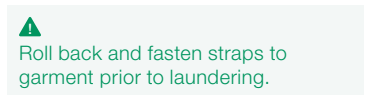Washing

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean

Drying

- Tumble dry low heat with other laundry items
- Do not iron
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls

Material content

Blend of: nylon (polyamide), polyurethane, polyester, spandex (elastane)

Not made with natural rubber latex.

Warranty

We guarantee your Tribute Wrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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Lohmann & Rauscher B.V
Lohmann & Rauscher Limited