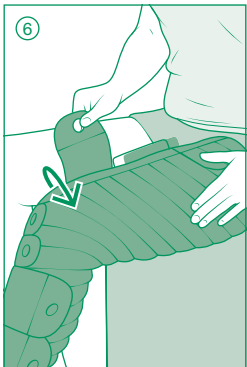
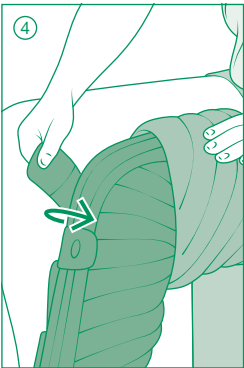
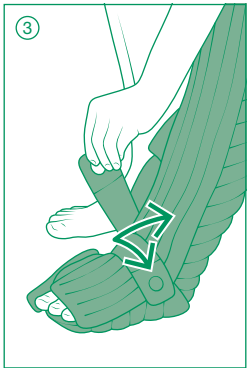
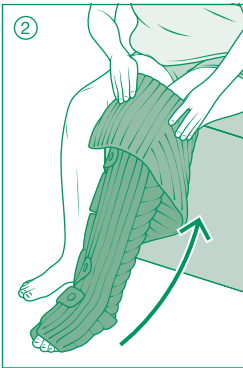
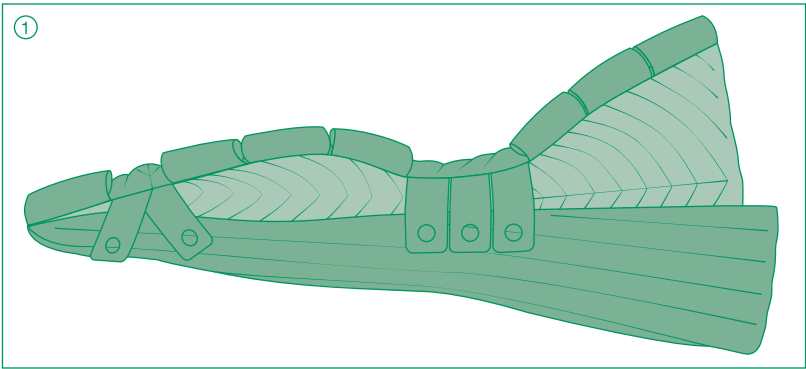
Tribute® Wrap Full Leg

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en Instructions for use

Thank you for choosing Tribute® Wrap!

Tribute Wrap Full Leg is an off-the-shelf adjustable foam gradient compression garment for the self-management of indicated conditions of the leg and foot during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Various stages of lipedema
- Edema

Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of various stages of lipedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue
- Help increase tissue oxygenation

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- ① Lightly fasten the two narrow ankle straps and three narrow knee straps. Loosely roll the remaining straps back and fasten to the garment.
- ② Fold the upper portion of the garment inside out overlapping the lower portion, then slide your foot into the wide end of the garment and pull it up to your knee.
- ③ Pull the two narrow ankle straps using the finger grip and fasten to the garment at an angle. Adjust for fit and comfort.
- ④ Pull the three narrow knee straps using the finger grip and fasten to the garment to contour around the knee. Adjust for fit and comfort.
- ⑤ Unroll and fasten the straps up to the knee starting closest to your toes. Adjust for fit and comfort.
- ⑥ Unfold and pull the upper portion of the garment up the leg and fasten the remaining straps beginning with the

top strap. Adjust for fit and comfort.

- ♦ You may prefer to complete Step 6 while standing.

- ⑦ Slide the wide end of the Sleep Sleeve over the garment and fasten it 2 inches (5 cm) below the top of garment.

Note: Illustration shown with Regular length garment. Other garment lengths have slightly different strap layout.

Care instructions

L&R recommends all Tribute Wrap garments be washed and dried before the first extended use.

Fasten straps to garment prior to laundering.

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean
- Tumble dry low heat with other laundry items
 - ♦ Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls
- Do not iron

Material content

Polyurethane, nylon (polyamide), polyester, spandex (elastane)
Not made with natural rubber latex.

Warranty

We guarantee your Tribute Wrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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Lohmann & Rauscher Limited