en Instructions for use for Tribute® Wrap Glove Adjustable foam compression wrap

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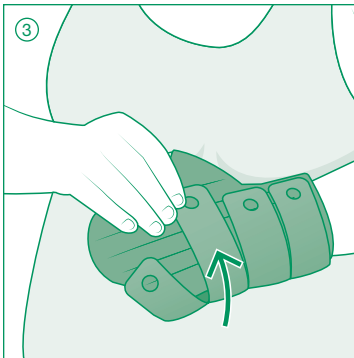
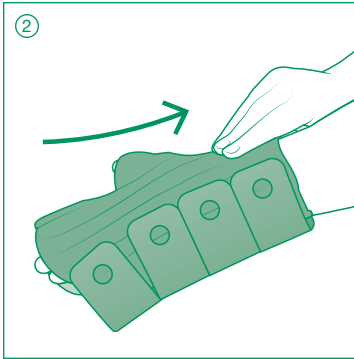
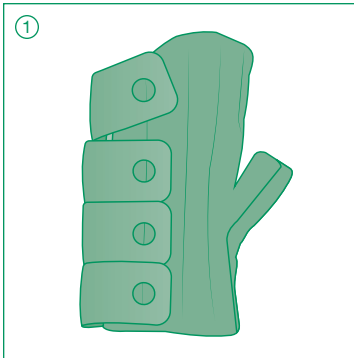
Tribute® Wrap Glove

en | Instructions for use

Thank you for choosing Tribute® Wrap!

Tribute® Wrap Glove is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the hand during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended Uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

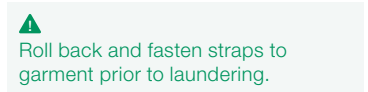
Donning instructions

- ① Fasten the narrow straps at the edge of the glove.
- ② Slide your hand into the glove ensuring your fingers align in the individual spacers and thumb in separate thumb space.
- ③ Pull the three straps closest to your wrist using the finger grip and fasten to the garment.
- ④ Pull the last strap and fasten to the garment at an angle. Adjust all straps for fit and comfort.

- ⑤ Slide the Sleep Sleeve over the glove leaving 1" at both ends of the glove and fasten.

Care instructions

L&R USA recommends all Tribute Wrap garments be washed and dried before the first extended use.

Washing

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean

Drying

- Tumble dry low heat with other laundry items
- Do not iron
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls

Material content

Blend of: nylon (polyamide), polyurethane, polyester, spandex (elastane)

Not made with natural rubber latex.

Warranty

We guarantee your Tribute Wrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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Lohmann & Rauscher B.V
Lohmann & Rauscher Limited