

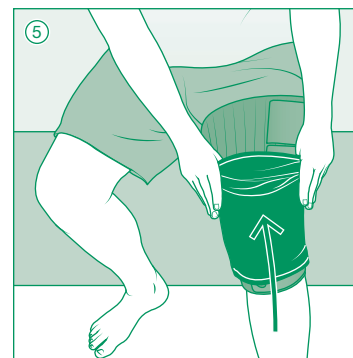
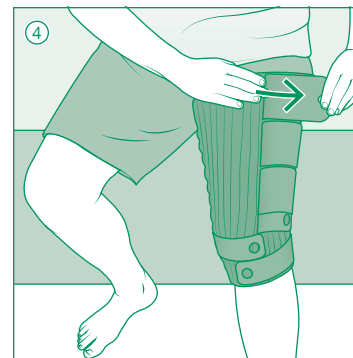
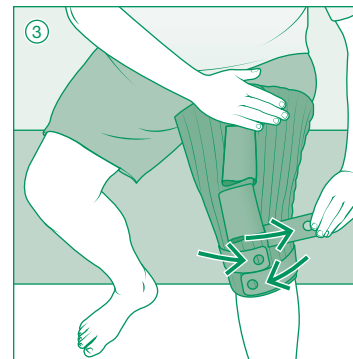


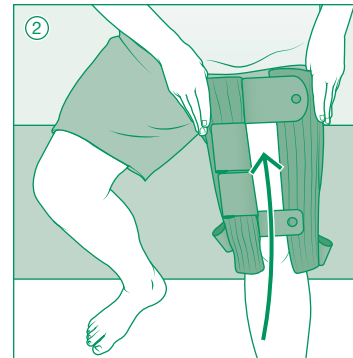

en | Instructions for use for
Tribute® Wrap Knee to Thigh
Adjustable foam compression wrap

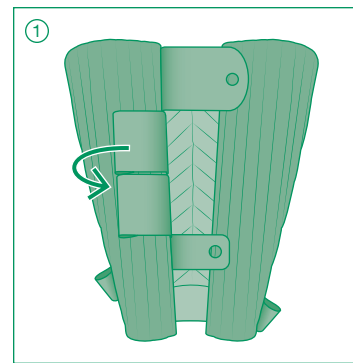

en | Instructions for use

Thank you for choosing Tribute® Wrap!

Tribute® Wrap Knee to Thigh is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the knee and thigh during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Various stages of lymphedema
- Edema

Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- ① **Fasten** the third narrow strap from the bottom and the final wide strap at the edge of the garment. Loosely roll the remaining straps back and fasten to the garment.
- ② **Slide** your foot into the wide end of the garment and pull it up your leg. Align the bottom of the garment just below the knee.
- ③ **Overlap** the two edges of the garment over top of the knee and using the finger grip, pull the narrow straps one at a time to conform around the knee. Adjust for fit and comfort.
- ④ **Fasten** the remaining straps using the same process, beginning closest to the knee. Adjust for fit and comfort.
- ⑤ **Slide** the wide end of the Sleep Sleeve over the garment and fasten it 2" below the top of the garment.

Note: Illustration shown with Regular length garment. Other garment lengths have a slightly different strap layout.

Care instructions

L&R USA recommends all Tribute Wrap garments be washed and dried before the first extended use.

- ▲ Roll back and fasten straps to garment prior to laundering.

Washing

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean

Drying

- Tumble dry low heat with other laundry items
- Do not iron
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls



Material content

Blend of: nylon (polyamide), polyurethane, polyester, spandex (elastane)

Not made with natural rubber latex.

Warranty

We guarantee your Tribute Wrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

Tribute® Wrap Knee to Thigh

Adjustable foam
compression wrap



Symbol glossary provided at: www.Lohmann-Rauscher.com/symbolsglossary

L&R USA INC.
3880 W Wheelhouse Road, Milwaukee, WI 53208 USA
www.Lohmann-Rauscher.com · 1-855-892-4140

EMERGO EUROPE
Prinsessegracht 20 · 2514 AP The Hague · The Netherlands

4040512 2049



US

L&R USA INC.
3880 W Wheelhouse Road
Milwaukee, WI 53208
USA

855-892-4140
inquiries@us.LRmed.com
www.Lohmann-Rauscher.us