**en** Instructions for use for Tribute® Wrap MCP to Axilla Adjustable foam compression wrap

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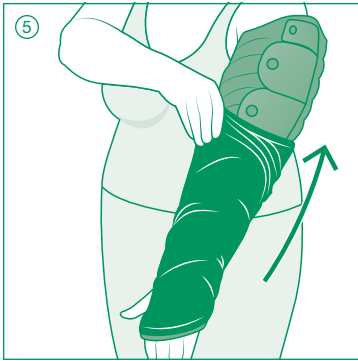
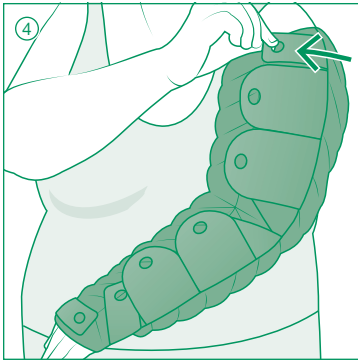
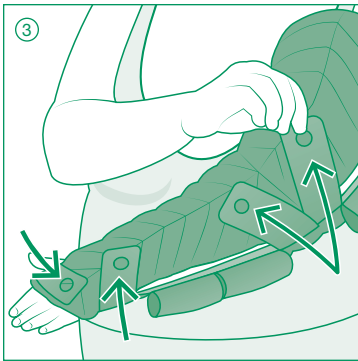
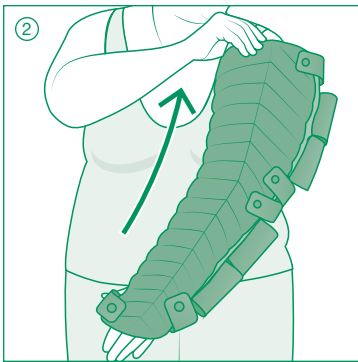
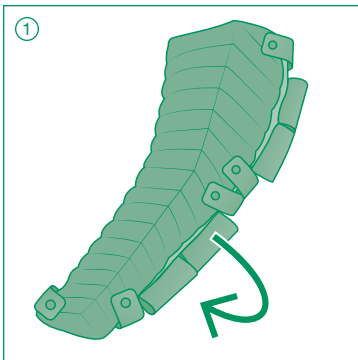
Adjustable foam compression wrap  
Adaptives Kompressionssystem aus Schaumstoff

## en | Instructions for use

### Thank you for choosing Tribute® Wrap!

Tribute® Wrap MCP to Axilla is an off-the-shelf adjustable foam garment for the self-management of lymphedema or edema of the arm during the evening, night, or low activity. Tribute Wrap comfortably adjusts to your unique shape to help maintain gains made during therapy, while allowing the flexibility to support further reduction.

Please consult a trained medical professional prior to first use.

**Indications and patient groups**

- Various stages of lymphedema
- Edema

#### Intended uses

- Helps maintain reduced volume and shape by supporting extremities
- Management of various stages of lymphedema
- Management of edema
- Assists in softening of indurated/fibrotic tissue

#### Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (e.g. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

#### Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment
- Any open wounds should be dressed prior to application

#### Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

#### Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

### Donning instructions

- 1 **Fasten** the narrow straps (and final wide strap for Long garments) at the edge of the garment. Loosely roll the remaining straps back and fasten to the garment.
- 2 **Slide** your arm into the wide end of the garment, ensuring your thumb is in the thumb space and elbow is centered between the two angled straps at the elbow.
- 3 **Pull** the straps at your hand, wrist, and elbow one at a time using the finger grip and fasten to the garment at an angle. Adjust for fit and comfort.
- 4 **Fasten** the remaining straps using the same process, beginning closest to the wrist.
- 5 **Slide** the wide end of the Sleep Sleeve over the garment ensuring your thumb is in the thumb space and fasten it 2" below the top of garment.

Note: Illustration shows Regular length garment. Other garment lengths have a different strap layout.

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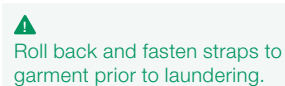
#### Washing

- Machine wash in cold water
- Do not use fabric softener or bleach
- Do not dry clean

#### Drying

- Tumble dry low heat with other laundry items
- Do not iron
- Shorten drying time by pressing out excess water with a towel or tumbling with clean tennis balls

#### Material content

Blend of: nylon (polyamide), polyurethane, polyester, spandex (elastane)

Not made with natural rubber latex.

#### Warranty

We guarantee your Tribute Wrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

#### Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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**Lohmann & Rauscher B.V**
**Lohmann & Rauscher Limited**