Tribute® Wrap Sleep Sleeve

Accessory for adjustable foam compression wrap
Zubehör für adaptives Kompressionssystem aus Schaumstoff
Accessoire pour système de compression en mousse réglable
Accesorio para venda semirrígida para terapia
compresiva ajustable de espuma
Accessorio per sistema compressivo regolabile con imbottitura
Accessoire voor verstelbaar schuimcompressiesysteem
Tilbehør til justerbar skumkompressionssystem

Symbol glossary provided at:
<https://www.lohmann-rauscher.com/label-symbols/>

L&R USA INC.
3880 W Wheelhouse Road
Milwaukee, WI 53208 USA
www.Lohmann-Rauscher.com

EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Lohmann & Rauscher AG
Oberstrasse 222
CH-9014 St. Gallen
Switzerland

UK Responsible Person
L&R UK Medical Ltd.
1 Wellington Court · Lancaster Park · Needwood ·
Burton upon Trent · Staffordshire · DE13 9PS ·
United Kingdom

4049452 2755 / 2023-07

en Instructions for use

Tribute® Wrap Sleep Sleeve is an accessory for use with the Tribute Wrap adjustable foam compression wrap.

Refer to Tribute Wrap garment instructions for use for indications, intended uses, contraindications, and precautions.

Please consult with a trained medical professional prior to first use.

Donning instructions

- ① Slide the Sleep Sleeve over the garment and fasten to the garment using the provided hook and loop fastener.

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Care instructions

- L&R recommends all Tribute Wrap accessories be washed and dried before the first extended use.
- Machine wash in cold water
 - Do not use fabric softener or bleach
 - Do not dry clean
 - Tumble dry low heat with other laundry items
 - Do not iron

Material content

Blend of: nylon (polyamide), spandex (elastane), polyester*, polyurethane*
*accessories with non-skid pad only
Not made with natural rubber latex.

Warranty

We guarantee your Tribute Wrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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Lohmann & Rauscher Limited 21 Queen Street Auckland 1010 New Zealand
Lohmann & Rauscher AB Grönegatan 1C 222 24 Lund Sweden