TributeNight™

Custom Lymphedema Garment for Evening & Nighttime Therapy

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Thank you for choosing TributeNight™!

TributeNight™ garments are therapeutic garments that provide lymphedema and edema therapy while you're at rest or sleeping. Designed by lymphedema therapists, TributeNight garments provide an easy to use solution as compared to nightly multi-layer bandaging which help you maintain your successful reductions or even help further reduce volume after therapy.

NE

TributeNight Benefits

- Delivers customized therapy for your unique presentation
- Provides maintenance therapy while you are at rest and helps further reduce volume
- Designed to be easy to don and doff, based on your ability
- Empowers you to manage your therapy without the need of a caregiver
- Increases compliance by helping save time and reduce stress
- Foam technology helps break up hardened, fibrotic skin
- Allows freedom of movement in the evening and throughout the night
- Easy to care for in washing machine and dryer

Rest assured. With TributeNight, your garment will work hard, allowing you to rest.

TributeNight garments are now powered by Coolcore®!

What is Coolcore?

Coolcore is an innovative fabric that rapidly wicks away sweat and body moisture to regulate your body temperature and keep you cool and comfortable.

What does Coolcore offer to you?

- Offers a lasting cooling effect with chemical-free technology that withstands wash and wear, while making it safe for you and the environment
- Resists pills and snags throughout the one-year life of the garment maintaining garment integrity
- Provides a comfortable experience with its soft feel
- Keeps your skin safe by not containing natural rubber latex

SURE-FIT™ Guarantee*

As part of our service to you, L&R USA ensures you have a properly fitting garment – guaranteed! To take advantage of our SURE-FIT™ Guarantee, please register your garment with L&R USA, and work with your medical professional/fitter for garment alterations.

Register your TributeNight using the included card, or online at: www.lohmann-rauscher.us/tributenight-register

NE

1

First Fit

1–45 days

To make sure your garment fits just right, we will alter or re-make your garment as many times as needed within the first 45 days from shipment.

2

Reduction Alteration

45 days–1 year

To support your successful limb reduction, we offer one free alteration during the one-year life of the garment to reduce the size of the garment to match your improved measurements.

3

Materials & Manufacturing

1 year

We guarantee your high-quality garment to be free of material and manufacturing defects during the recommended one-year life of the garment.

*Valid on garments for patients in North America only

Care Instructions

EN

- L&R USA recommends that all TributeNight garments be washed and dried before the first extended use.
- TributeNight garments can be washed and dried with your other laundry.

Washing

- Machine wash on permanent press in warm water
- Do not use bleach or fabric softener
- Take extra care if the garment has VELCRO® brand fasteners and ensure it is fastened securely prior to washing
- Do not dry clean

Drying

- Tumble dry on permanent press with other laundry items or clean tennis balls
- Drying time may be shortened by pressing out excess water with a towel prior to drying
- Do not iron

Material Content




- ◆ Polyurethane, polyester, spandex/elastane
Accessory information available upon request.
- ◆ Not made with natural rubber latex

Donning Instructions

We have provided donning instructions for an overview of common styles. Given TributeNight is completely custom, not every garment style or modification has been included.



If you have any questions about donning your TributeNight after reading the instructions below, please reach out to our Custom Design Center from 7 AM–7 PM Central at 414-892-5158 or customdesigncenter@us.LRmed.com.

Arm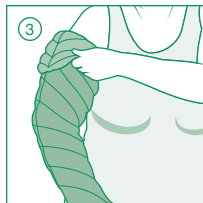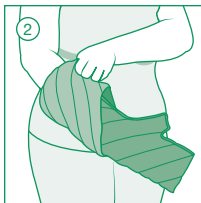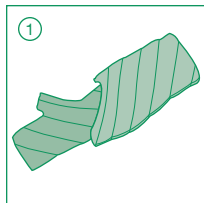




1. **Fold** the upper portion of the garment over the lower portion.
2. **Slide** the folded garment over your hand and lower arm ensuring that the seam is faced toward your body.
3. **Unfold** the garment onto your upper arm. Adjust for comfort.

Below Knee/Thigh High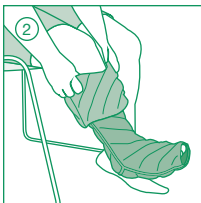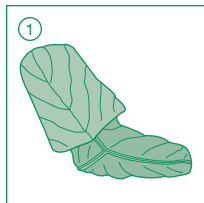





1. Fold the upper portion of the garment over the lower portion.
2. Pull the garment on over the foot until the foot and heel are aligned in the foot portion of the garment.
3. Unfold the garment over your ankle and slide onto your calf/thigh. Adjust for comfort.

Pants/Shorts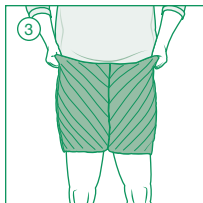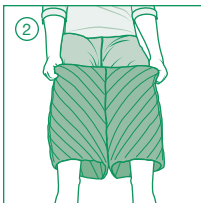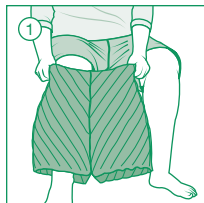

1. Place your desired first leg into one of the legs of the garment.
2. Place your other leg into the opposite leg of the garment.
3. Pull the garment up to the waist. Adjust for comfort.

Mandibular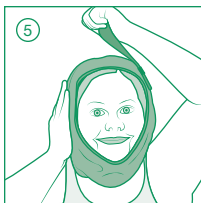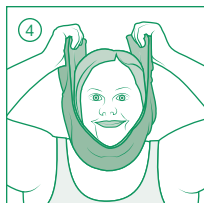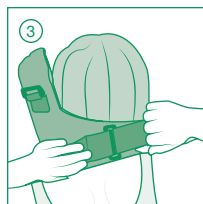
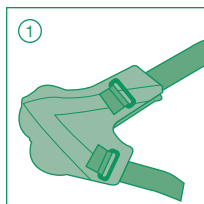


1. **Unfasten** and remove both straps from their buckles, noting the short strap will be fastened at the back of neck and the long strap at the top of the head.
 - ◆ Small white tags are sewn into the end of each strap below the VELCRO® brand hook fastener for clarification.
 - ◆ Fold the VELCRO® brand hook fastener back onto itself so it does not stick to the garment or your clothing.
2. **Hold** the lower portion of the garment on opposite sides and place the garment around your neck underneath your chin.
3. **Hold** the buckle side of the garment against your neck, thread the short strap through the buckle opening, then pull the strap back over the top of the buckle, fastening the VELCRO® brand hook closure onto the garment lightly. Adjust for comfort.

Mandibular, continued






4. **Align** the garment to contour the neck and face. Adjust its position using the two ends of the garment near the top of the head.
5. **Thread** the long strap through the buckle opening while holding the buckle side of the garment against the top of your head. Pull the strap back over the top of the buckle, fastening the VELCRO® brand hook closure onto the garment. Adjust for comfort.

Shirt/Vest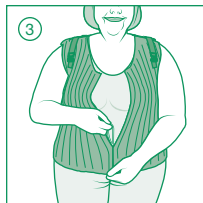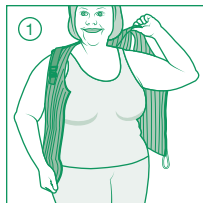

1. **Place** your desired first arm into one of the arms of the garment.
2. **Place** your other arm into the opposite arm of the garment.
3. **Close** the garment completely with the zipper, starting from the lower portion of the garment up to the end of the zipper track. Adjust for comfort.

Body Suit

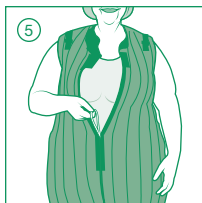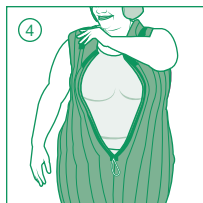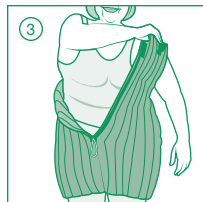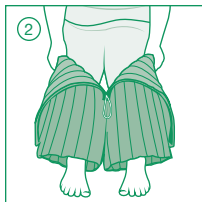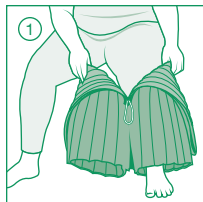1. Place your desired first leg into one of the legs of the garment.
2. Place your other leg into the opposite leg of the garment.
3. Pull the upper portion of the garment up your body and place your desired first arm into the appropriate arm of the garment.
4. Place your other arm into the opposite arm of the garment.
5. Close the garment completely with the zipper, starting from the lower portion of the garment up to the end of the zipper track. Adjust for comfort.

Indications

- Various stages of lymphedema
- Edema (non-blood related)

Intended Uses

- Helps maintain reduced volume and shape by supporting extremities and torso
- Management of various stages of lymphedema
- Management of edema (non-blood related)
- Assists in softening of indurated/fibrotic tissue

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Untreated congestive heart failure
- Untreated infection (e.g., cellulitis)
- Allergies or sensitivities to materials

Precautions

- Impaired sensation (peripheral neuropathy)
- Cognitive impairment

 TributeNight garments are manufactured in the USA.

Custom Design Center

Our Custom Design Center is available to answer your questions, such as: how to don your garment, how to register your garment, or garment care.

Available Monday–Friday, 7:00 AM–7:00 PM Central

Phone: 414-892-5158

Email: customdesigncenter@us.LRmed.com

Website: www.Lohmann-Rauscher.us

Outside the US, please visit www.lohmann-rauscher.com

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Coolcore® is a registered trademark of Coolcore, LLC.
VELCRO® is a registered trademark of Velcro BVBA.