

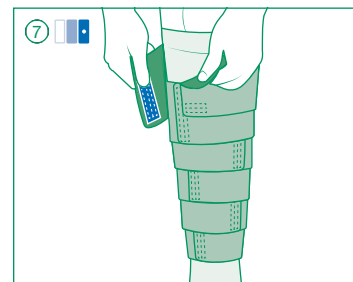
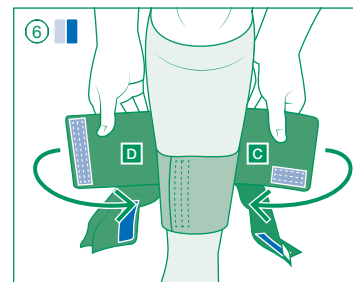
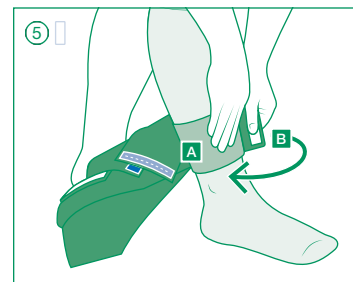
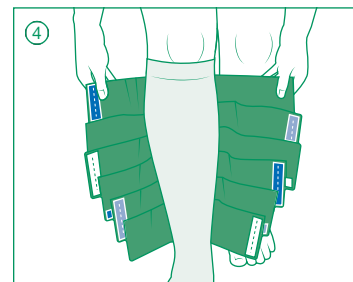
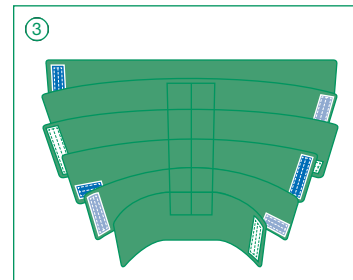


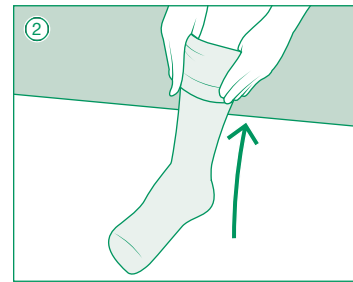

en Instructions for use for
ReadyWrap® Fusion Kit
Adjustable compression wrap plus
liner with compressive foot and ankle

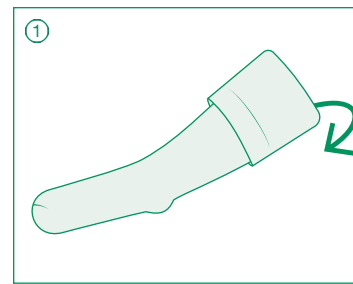

en | Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap® Fusion Kit brings together the ReadyWrap Calf adjustable compression wrap with a built-in 50% overlap and muscle support spine to contain the calf and ReadyWrap Fusion Liner below knee liner with a compressive foot and ankle for a complete lower leg solution designed for self-management of indicated conditions.

Please consult a trained medical professional prior to first use.

Indications

- Lymphedema
- Edema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- ① **Fold** the top of the Fusion Liner inside-out one-third so it overlaps the lower portion of the garment.
- ② **Slide** your foot into the liner and pull it on over the heel. Continue pulling the liner the rest of the way up the leg a few centimeters at a time ensuring it lays flat without creasing.
- ③ **Unfasten** all garment straps.
- ④ **Align** the spine at back of calf with the hook side facing your calf.

⑤ **Hold** the shortest strap set in opposing hands just above the heel, and place the strap without hook (A) on the front of your leg about 2–3 cm above your ankle bone. While holding that strap down, pull the opposing strap with the long vertical white hook (B) at full stretch and fasten it directly on top of the strap below. (Strap set 1)

⑥ **Fasten** the next set of straps by applying the strap with the short horizontal light blue hook (C) at a full stretch onto the straps below. Next, pull the opposing strap with the long vertical light blue hook (D) at full stretch and fasten it directly on top of the strap below. (Strap set 2)

⑦ **Fasten** the remaining straps in sequence using the same process until all straps are in place.

- ◆ Regular length calf garments have 5 strap sets. Long length calf garments have 6 strap sets.

Care instructions

L&R USA recommends all ReadyWrap garments be washed and dried before the first extended use.

Calf

- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
 - ◆ Be sure that the VELCRO® brand fasteners do not contact your towel.
- Do not iron
- Do not dry clean



Liner

- Machine wash in cold water
- Do not use fabric softener or bleach
- Lay flat to dry
- Do not iron
- Do not dry clean



Material content

Calf: Nylon (polyamide), polyurethane, spandex (elastane)
Not made with natural rubber latex.

Liner: Nylon (polyamide), spandex (elastane)
Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garments to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

ReadyWrap® Fusion Kit

Adjustable compression wrap plus liner with compressive foot and ankle

us

L&R USA INC.
3880 W Wheelhouse Road
Milwaukee, WI 53208
USA

855-892-4140
inquiries@us.LRmed.com
www.Lohmann-Rauscher.us

4041230 2219



Symbol glossary provided at: www.Lohmann-Rauscher.com/symbolsglossary