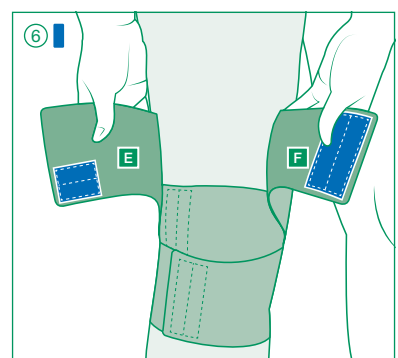
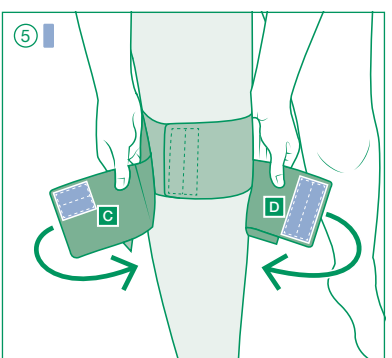
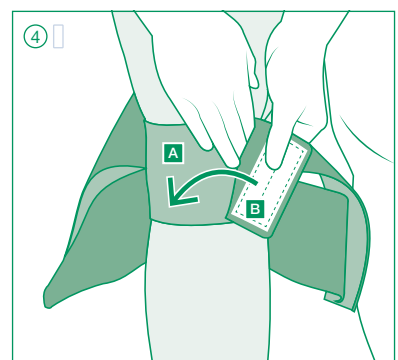
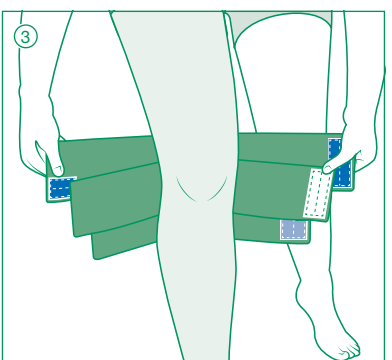
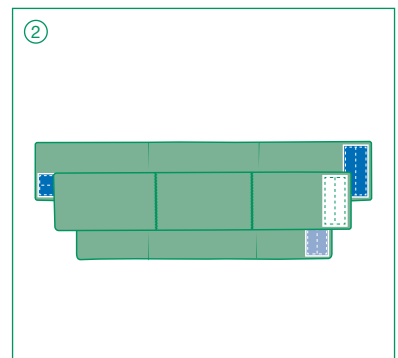
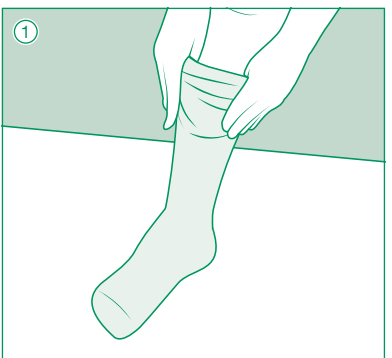
ReadyWrap® Knee

Adjustable compression wrap
Adaptives Kompressionssystem
Système de compression réglable
Sistema de compresión ajustable
Sistema compressivo regolabile

Verstelbaar compressiesysteem
Adaptívny kompresívny systém
Uyarlanabilir kompresyon sistemi
Justierbar kompressionssystem
الضاغط والقابل للضبط والتعديل

en Instructions for use

Thank you for choosing ReadyWrap®!

ReadyWrap is a low-stretch adjustable compression wrap designed for self-management of indicated conditions. ReadyWrap Knee has adjustable straps with a built in 50% overlap to contain the knee, while allowing movement. All ReadyWrap styles can be worn during the day and/or night.

Please consult a trained medical professional prior to first use.

Indications and patient groups

- Lymphedema
- Lipedema
- Edema
- Venous ulcers
- Venous insufficiency
- Leg fatigue and/or tiredness

Intended uses

- To maintain reduced fluid volume and shape of limbs
- Management of lymphedema, lipedema and edema
- Relief of leg fatigue and heaviness
- Assist in management of some symptoms of venous ulcers
- Assists in helping limit venous ulcer recurrence
- Compression garments for phlebological, lymphological indications (i.e., venous disease, chronic edema and lymphedema)

Contraindications

- Arterial insufficiency or ABPI < 0.8
- Moderate to severe peripheral arterial disease
- Acute DVT
- Untreated congestive heart failure
- Untreated cancer
- Untreated infection (i.e. cellulitis)
- Known allergy and/or hypersensitivity to any of the product components
- Severe cognitive impairment

Precautions

- ABPI > 1.3
- Absent or impaired sensation (peripheral neuropathy)
- Any open wounds should be dressed prior to application
- Cardiac overload is suspected
- Patients have diabetes
- Patients have advanced small vessel disease
- Arterial disease is present
- Renal failure is present
- Rheumatoid arthritis is present

Side effects

In (very) rare cases, skin irritations and/or allergies may occur.

Serious incident

In case of serious incident, contact the manufacturer and the responsible health authorities.

Donning instructions

- 1 toe
- 2 foot
- 3 calf
- 4 thigh
- 5 **knee**

1 Slide the ReadyWrap Liner onto your leg.

2 Unfasten all garment straps.

3 Align the middle strap set at back of knee with hook side facing toward your knee.

4 Place and hold the middle strap

without hook (A) directly over your knee. Pull the opposing strap with white hook (B) at full stretch and fasten it directly on top of the strap below. (Strap set 1)

5 Fasten the strap set with the short horizontal light blue hook (C) below the knee at full stretch and fasten to the straps below. Next, pull the opposing strap with the long vertical light blue hook (D) at full stretch and fasten it directly on top of the strap below. (Strap set 2)

6 Fasten the strap set with dark blue hook (E, F) above the knee using the same process. (Strap set 3)

Care instructions

- L&R recommends all ReadyWrap garments be washed and dried before the first extended use.
- Hand wash in warm water with mild detergent, rinse thoroughly in warm water
- Do not use fabric softener or bleach
- Do not dry clean
- Lay on a towel and squeeze your ReadyWrap to remove excess water, lay flat on a dry towel to finish drying
- Be sure that the VELCRO® brand fasteners do not contact your towel
- Do not iron

Material content

Nylon (polyamide), polyurethane, spandex (elastane)
Not made with natural rubber latex.

Warranty

We guarantee your ReadyWrap garment to be free of material and manufacturing defects during the recommended six month life of the garment.

Disposal

Observe current national regulations and laws for disposal of this device. Packaging marked recyclable should be properly recycled, otherwise disposed of with household waste.

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