
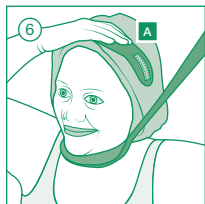
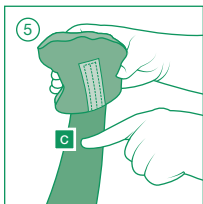
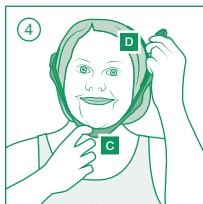
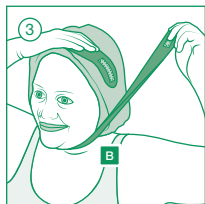
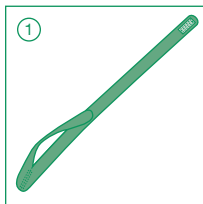
SwellSpots®

Head-Neck Pad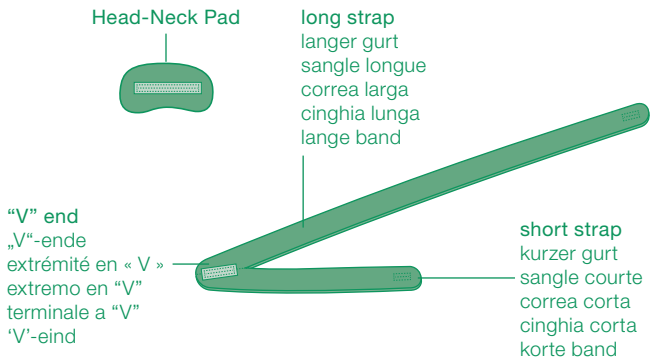

Applying your Head-Neck Pad

1. Fasten the short strap to the long strap using the VELCRO® brand hook closure, creating a loop.
2. Place the loop over the crown of your head so the “V” end is over the desired ear (A), and the long strap is across the opposite ear (B).
3. Draw the long strap under your chin (B) and fasten to the hook-receptive fabric over your ear.
4. Hold the long strap at the desired position of your Head-Neck Pad (C). Unfasten the long strap (D) while continuing to hold the location for the Head-Neck Pad.
5. Lay the straps on a flat surface and attach the Head-Neck Pad to the long strap at the desired location (C).
6. Place the loop over the crown of your head and refasten the long strap to the “V” end over your desired ear (A). Adjust for fit and comfort.

Removing your Head-Neck Pad

- Unfasten the long strap (D) from over your ear and take the unit off of your head.
- Leave your Head-Neck Pad attached to the long strap for the next time you apply your garment.

Re-applying your Head-Neck Pad

- Place the loop over the crown of your head so the “V” end is over the desired ear (A), and the long strap is across the opposite ear (B).
- Draw the long strap under your chin (B) and fasten to the hook-receptive fabric over your ear.

- View the Head-Neck Pad application video by visiting: www.Lohmann-Rauscher.us/swellspots/
- To make application easier, you may want to stand in front of a mirror
- Can be worn over left or right ear — application over left ear shown

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